


E-Z ON LATERAL SUPPORT

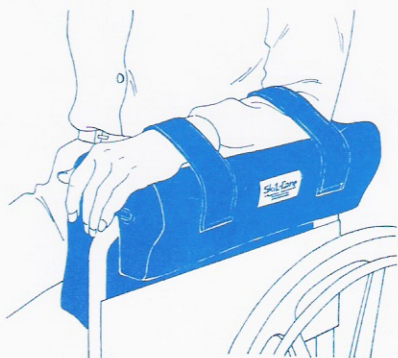

Reorder

□706005

Purpose

Skil-Care's E-Z On Lateral Support provides restraint-free positioning. It controls lateral leaning by stabilizing the torso. This device may be used on both the wheelchair and gerichair.

Application

The E-Z On Lateral Support may be used on either the right or left side of the chair.

1. Place the device over wheelchair or gerichair armrest. (Illustration A)

2. Seat the resident as far back in the chair as possible and place his/her arm in the trough.

3. The E-Z On Lateral Support may be used without the arm straps (Illustration B), or with the straps (Illustration C) if the patient requires them in order to keep his/her arm securely positioned in the trough.

Easy to Care For

The nylon cover wipes clean with spray-and-wipe cleanser. A non-chlorine disinfectant may be used.

