


Position Plus+™

Cushion and Wedge

*Forward sliding and lateral leaning must be controlled to achieve proper positioning. **Position Plus+ Cushions and Wedges** are designed with four gel-filled pods that act independently to keep the wheelchair user centered on the seat. These comfortable cushions and wedges reduce interface pressure, eliminate sling-seat hammocking, and prevent skin-damaging heat build-up.*

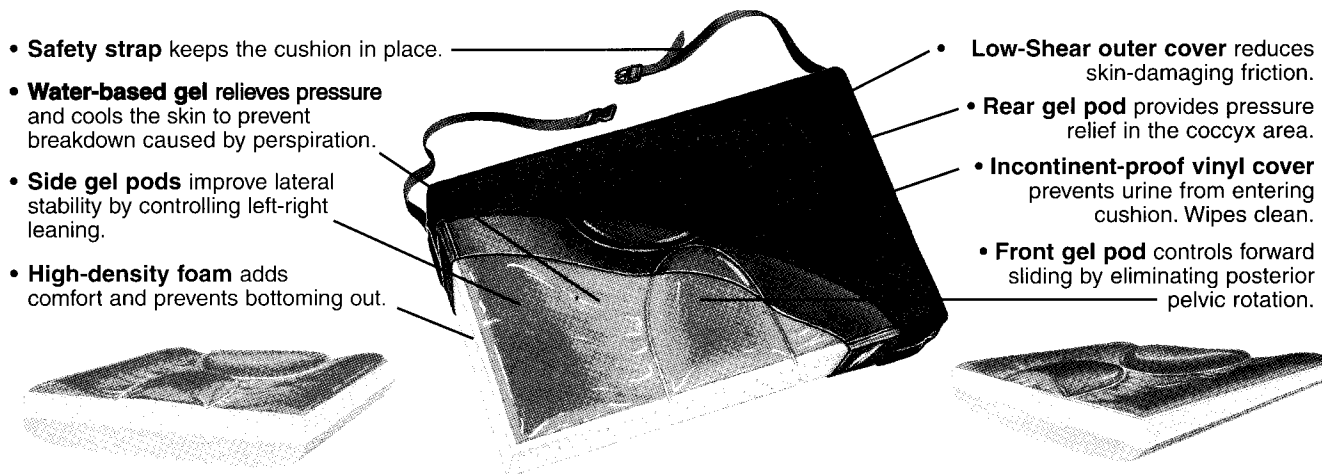

- **Safety strap** keeps the cushion in place.
- **Water-based gel** relieves pressure and cools the skin to prevent breakdown caused by perspiration.
- **Side gel pods** improve lateral stability by controlling left-right leaning.
- **High-density foam** adds comfort and prevents bottoming out.

- **Low-Shear outer cover** reduces skin-damaging friction.
- **Rear gel pod** provides pressure relief in the coccyx area.
- **Incontinent-proof vinyl cover** prevents urine from entering cushion. Wipes clean.
- **Front gel pod** controls forward sliding by eliminating posterior pelvic rotation.

Position-Plus+ Cushion

Contoured base reduces sling-seat hammocking. It rests below frame so user sits lower in chair.

Position-Plus+ Wedge

Contoured wedge base controls forward sliding and reduces sling-seat hammocking. It rests below seat frame so user sits lower in chair.

Important Safety Notice

The Position Plus+ Cushions and Wedges have safety straps that must be secured to the wheelchair to prevent them from sliding off the seat.

Cover Care Instructions

The vinyl inner cover may be cleaned with a damp cloth. Spray-type cleaners and mild disinfectants may be used. The cloth outer cover is completely launderable. Do not launder or dry the cloth cover at temperatures that exceed 180°F (82°C). Do not use bleach.

Position Plus+ Slide-Guard Cushion

Reorder#	Size (Inches)	Reorder#	Size (Inches)	Reorder#	Size (Inches)	Thrust Barrier Height
<input type="checkbox"/> 751420	16 x 16 x 21/2	<input type="checkbox"/> 751435	18 x 16 x 21/2	<input type="checkbox"/> 751450	20 x 16 x 21/2	1

Position Plus+ Slide-Guard Wedge

Reorder#	Size (Inches)	Reorder#	Size (Inches)	Reorder#	Size (Inches)	Thrust Barrier Height
<input type="checkbox"/> 751465	16 x 16 x 31/2 x 1	<input type="checkbox"/> 751480	18 x 16 x 31/2 x 1	<input type="checkbox"/> 751495	20 x 16 x 31/2 x 1	1