



**GEAR UP
FOUNDATION**

Endorsed and used by GearUp Foundation Globally
<https://www.gearupworld.org>



Storage Bag



Re-Order #	Description	Size	U/M
914820	Emergency EVAC Transporter with Storage Bag	34"W x 72"L	Each
914824	Male Component for Positioning Straps	N/A	6 Each

Purpose

The SkiL-Care Emergency EVAC Transporter is a portable compact transport unit used to transport, transfer or rescue individuals from areas inaccessible to stretchers and/or bulkier devices. Constructed out of heavy-duty strength nylon and reinforced webbing the unit comes with six (6) ergonomic rubber handles for lifting and three (3) tear resistant heavy-duty locking straps to hold the injured person in place. The unit easily folds and can be place in the storage bag for convenience and quick accessibility.

Instruction

It is always advised that caregivers allow sufficient personnel for the lifting and carrying procedure. This is of course dependent on the injured persons weight, conditions (ie stairs or level ground) and the distance to be carried. These parameters should be determined during a training session prior to the actual use. The three positioning straps use high density plastic buckles, they can be replaced (Model 914824) Fig. 1. When replacing, follow the threading pattern with the other belts but always test before use.

Important Safety Information

Before each use, inspect the product for rips or tears, if present, replace the product immediately. Place the person in the transporter. Close the three upper locking straps and adjust for a snug fit. Never attempt to lift more weight than the recovery person can safely handle. The transporter can accommodate up to eight lifters with a max weight capacity of 500lbs. For repeat use follow the below cleaning instructions.

Cleaning Instructions

The transporters can may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use chlorine bleach



Fig. 1