



| Re-Order # | Description | Size | Unit |
|------------|---------------------------------|-----------|------|
| 252220 | Transfer Belt with Metal Buckle | Universal | Each |


Purpose

Safely assist patients or loved ones with ease using our transfer belt. The 5 padded handles allow for support from any angle, reducing the risk of caregiver injury and preventing falls.

Easy to Use

Place the belt around the patient with the buckle conveniently located in the front and the blue handles are on the back of the patient. After the belt is placed around the patient it should be adjusted to a snugness where the belt doesn't slip but isn't uncomfortable. Figure 1 shows the proper closing of the metal slide lock buckle.

Fig. 1

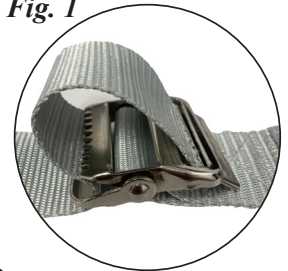

Once the belts are properly adjusted on the patient then based on the patient's weight and caregiver strength, a nursing professional should determine the appropriate number of aids to be used when assisting. Figure 2 illustrates that when the caregivers hand is placed in one of the handles, the hand should be placed palm up.

Fig. 2



Easy Care

The transfer belt can be laundered in an institutional laundry but dried at temperatures (180°F, 82°C or lower). The belts can also be wiped clean with a standard disinfectant. When the belt exhibits excessive wear, it should be discarded and replaced.