


## SUPER-SLINGS

Reorder #	Size	Description	Weight Capacity
☐ 914123	9" x 20"	2 - Handle (For use by one-two caregivers)	200 lbs
☐ 253020	36" x 40"	4 - Handle (For use by two caregivers)	250 lbs.
☐ 253010	46" x 46"	8 -Handle (For use by four caregivers)	500 lbs.
☐ 253030	36" x 72"	12 - Handle (For use by six caregivers)	500 lbs.

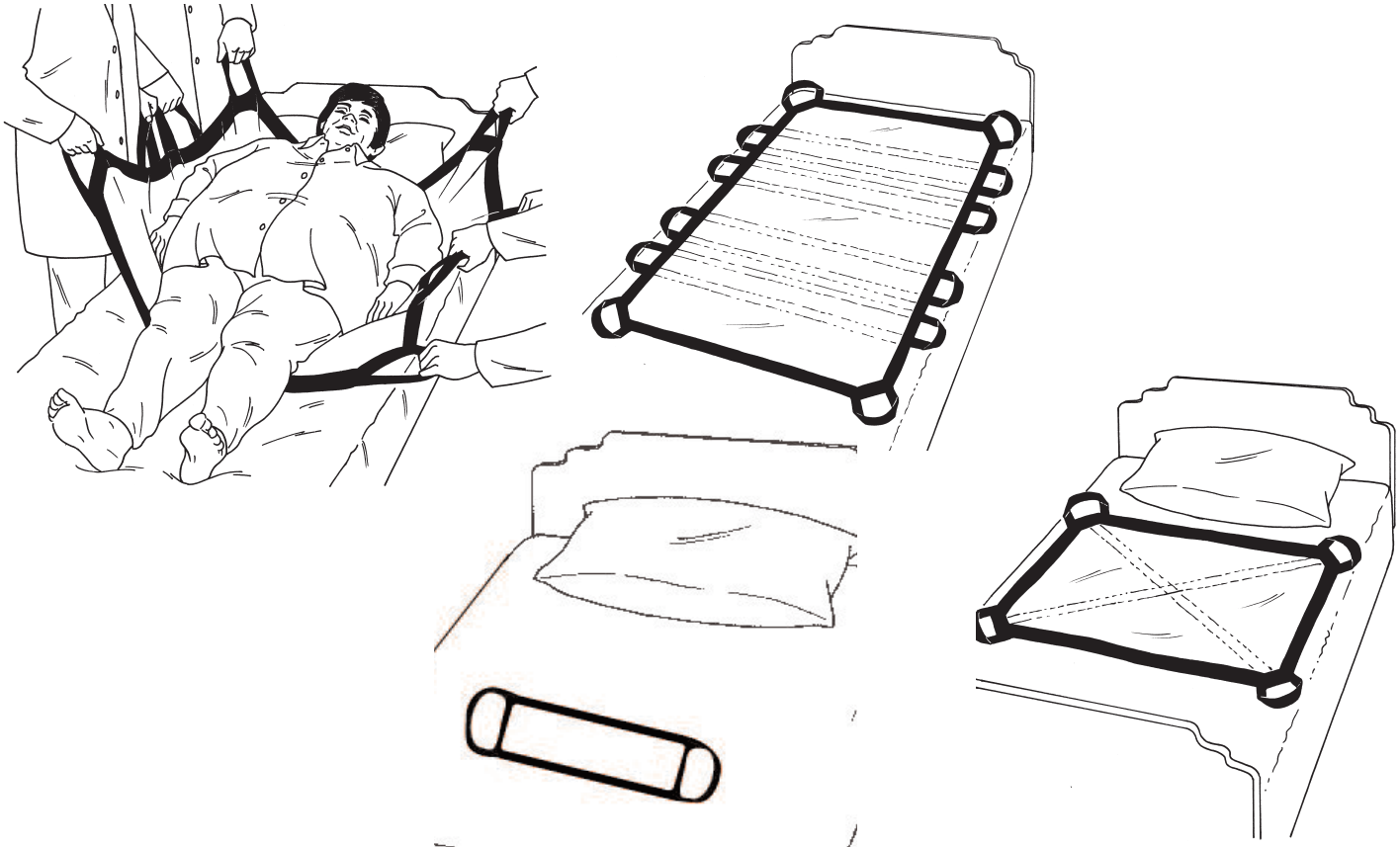

### Purpose

Super-Slings are designed to facilitate patient transfer, lifting, and positioning. They may also be used for emergency evacuations when wheelchairs or gurneys are either unavailable or not practical. Super-Slings reduce exposure to back injury by permitting multiple caregivers to lift, move, transfer and position patients. Skil-Care's Super-Slings are made from low-friction nylon that enables caregivers to slide patients easily on all surfaces, including beds and gurneys.

### Important Notices

1 Determine how many caregivers will be required to lift, carry, or move a specific patient.

1 Be certain to follow your facility's established policies for lifting, carrying, moving, or positioning patients.

1 Make certain that the patient's head is supported (either by positioning the Super-Sling beneath the head or by having a caregiver support the head with his/her hands) when lifting or carrying a patient.

### Easy Care

All Super-Slings may be washed and dried at temperatures that do not exceed 180°F (180°C). Do not use bleach.