



Re-Order #	Description	Size	Units
400110	Foam Head Positioning Cradle	8.5"L x 8"W x 4"H	6/PK

Purpose

When a patient must lie face down during a procedure, it can be uncomfortable and difficult to remain still. The Skil-Care Head Positioning Cradle provides support to help maintain a stable position for better comfort and outcomes.

Use Instructions

Remove the cushion from the package and place it under the patient's face while lying face down. Position the cushion so the nose rests in the base of the "T" and the chin in the top of the "T." Oxygen tubing may be routed through the side "V" cut for convenience.

Easy Care

This foam positioner is intended for single patient use. Do not wash.