



Re-Order Number	Descriptions	Size	Units
554022	30° Bed Wedge w/LSII Cover	7" x 12" x 16"	Pair
554025	30° Bed Wedge w/LSII Cover	7" x 12" x 24"	Pair
554035	30° Bed Wedge w/Vinyl Cover	7" x 12" x 24"	Pair
554036	30° Bed Wedge w/Vinyl Cover	7" x 12" x 34"	Pair
914451	30° Bed Wedge w/Convuluted surface w/LSII Cover	7" x 12" x 24"	Each
554030	30° Bed Wedge with High Density Foam w/LSII Cover - Bariatric	7" x 12" x 24"	Pair
554026	30° Bed Wedge w/LSII Cover	7" x 12" x 34"	Pair
554028	30° Bed Wedge with High Density Foam w/LSII Cover - Bariatric	7" x 12" x 34"	Pair
914452	30° Bed Wedge w/Convuluted surface w/LSII Cover	7" x 12" x 34"	Each

Purpose

Skil-Care's 30-Degree Positioning Wedge is designed to comply with the Agency for Healthcare Research and Quality (AHRQ) guidelines that recommend a 30-degree angle of incline as optimal for relieving pressure on the hipbone and coccyx.

Easy to Use

1. Place patient on his/side and slip tapered edge of wedge beneath patient, just below the shoulders. (Non-slip matting faces bed. It does not come in contact with patient).
2. Allow patient to roll back onto the wedge.
3. Reposition patient according to your facility's established procedures and policies.

Easy Care

The 30-Degree Positioning Wedge may be wiped clean using a spray-and-wipe cleanser. Disinfect using a non-chlorine disinfectant.