

Re-Order#	Size	Unit
554110	8" x 24'	Each
554120	6" x 18"	Each

Purpose

Available in two sizes, the versatile Ultra-Soft Bed Bosters provide positioning, pressure relief, and comfort. They may be used to secure the patient in a side-lying position, elevate heels, separate knees, among many other applications. Covered in low-shear Cozy Cloth, these bolsters protect delicate skin from abrasion, shear, and friction.

Suggested Uses:

Model 554110 (Size 8" x 24")



Place in front of patient to provide secure side-lying positioning.




Place in front of patient to control forward rolling.

Easy Care

The Cozy Cloth outer removes for laundering. It may be washed and dried at temperatures that do not exceed 180°F(84°C). Do not use bleach. The inner cover may be cleaned with spray-and-wide cleanser and disinfected with a non-chlorine disinfectant.

Notice: Launder only the outer cloth cover. Do not place bolster in washer or drier.

Model 554120 (Size 6" x 18")

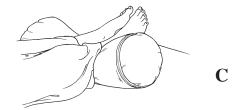

Place beneath ankles to off-load heels for pressure relief that prevents pressure sores form forming and aids in healing existing sores.




Place between to decrease leg and lower back pain and relax the spine.



Place between knees to control leg abduction, reduce skin shear, and promote spinal alignment.



Place beneath the neck to improve neck-spine alignment and increase sleeping comfort.

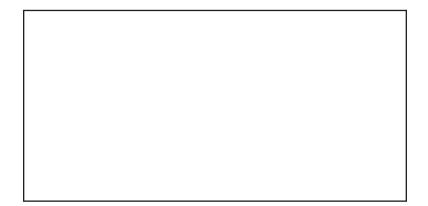

Manufactured by: Skil-Care Corporation • Made in the U.S.A

