



ReOrder	Description	Size	U/M
554130	Contour Leg Cradle	12"L x 8"D x 5"H	Each

Purpose

The Contour Leg Cradle is made of high-density foam and comes with our low shear II anti-bacterial fluid proof, wipe clean cover. The LSII two-way stretch material decreases skin shear forces. The bottom portion of the cover has a skid resistant surface to decrease movement during exercise/ROM. The contour cradle is designed to be placed under the knee (supine or prone), ankle or upper extremity.

Reason for a cradle

The concave top, cradles the extremity in order to decrease rotation and maintain the limb position. The two-way stretch outer cover allows for movement while reducing shear forces.

The cradle can be used to elevate the leg for increased blood flow, reduced swelling, lower leg pressure and help in preventing blood clots. Note: Example of a useful exercise when elevating the heel are ankle pumps.

Easy Care

The antibacterial fabric can be easily wiped cleaned with disinfectant wipes / liquids or soap and water. Do not launder!

