



Re-Order #	Description	Size	Units
554135	Foam Bolster	5"D x 16"L	5/PK

**Purpose**

Proper positioning of a resident is critical to maintaining comfort and preventing pressure sores. This universal bolster can be used under the neck, knee, arm or on the side of the individual.

**Use Instructions**

As mentioned previously, this positioning bolster can be used under the arm,knee, neck or on the side. Note that this bolster isn’t intended for stabilizing or preventing the resident from rolling off the bed.

**Easy Care**

The foam bolster can be subjected to limited laundering as long as it is placed in a laundry bag while maintaining a temperature less than 150°F (66°C).