



ReOrder	Description	Size	U/M
554150	U-Turn Bolster	20"W x 19"H	Each

Purpose

The U-shaped base helps a single care giver turn the patient to side lying with less exertion and stress. The curved interior slots keep the lower extremities separated and in a stable position. The flat sides allow the patient to stay in side lying so clinicians can check for wounds, change absorbency pads or for temporary re-positioning. The U-Turn is covered in our Low Shear II wipe clean material.

- Easy to operate; reduces the burden on families and caregivers.
- Medical Grade Exterior is made of high-quality waterproof wipe clean fabric, which is comfortable, hypoallergenic, and easy to clean.
- The interior is a high-density foam, the foam is soft, supportive, and long-lasting provide perfect feel of comfort and support.
- U-Shaped Arc Design: Allows for easy rotation with decreased force exertion from the clinician.

Easy to Use

With the patient on the bed, place the turning device under the patient's leg/knee area, with a hand holding the handle, and the other hand holding the patient's waist or back area turn the patient's body towards you. To hold the patient in a turned position, slide the strap through the handle and secure the strap to the guardrail to stabilize the patient's body.