



ReOrder	Description	Size	U/M
554150	U-Turn Bolster	20"W x 19"H	Each

Purpose

U-Turn Bolster allows doctors, nurses, friends, or family to turn a patient from their back to their side to perform wellness checks and daily hygiene.

- Made of high-quality foam with a skin friendly wipe clean cover.
- Allows one person to move a patient from back to side.
- Ideal for use at home or in a professional setting.
- Separates patient's legs for optimal visibility during safety checks.
- Straps allow the patient to stay in a side-lying position while clinicians can check for wounds.
- One size is suitable for everyone.

Easy to Use

With the patient on the bed, place the turning device under the patient's leg/knee area, with a hand holding the handle, and the other hand holding the patient's waist or back area turn the patient's body towards you. To hold the patient in a turned position, slide the strap through the handle and secure the strap to the guardrail to stabilize the patient's body.