

Cozy Cloth



Nylon Bottom



| Re-Order # | Description                               | Size        | Unit |
|------------|---|-------------|------|
| 556047     | Cozy Cloth & Nylon Slider Sheet w/Handles | 54"L x 40"W | Each |

### Product Description

This low friction Nylon slider sheet has continuous handles and a Cozy Cloth top surface which is ideal for repositioning and sliding the resident.

### Application

1. Place the slide sheet under the resident utilizing the "Log Roll" or similar technique.
2. Position the slide sheet completely under resident.
3. Always make sure the casters on the bed/gurney are locked during the repositioning of the resident. One caregiver on each side will grasp the handles of the underside of the sheet and gently pull the sheet towards the head of the bed.

### Easy Care

This slider sheet can be laundered and dried at temperatures that do not exceed 180°F (82°C).