



Re-Order #	Description	Size	Unit
556085	Smartline Chair Glide with one-way locking	17"L x 17"W	Each

Purpose

Workforce safety and safe patient handling are critical in today's environment. Positioning patients puts the staff at risk of musculoskeletal disorders, including low back pain, sciatica and rotator injuries. In order to reduce the risk of injury the Skil-Care team has developed the Chair Glide. The Chair Glide is designed to stay under the patient for an extended period and is always ready to assist in boosting and repositioning.

Easy To Use

1. Place the Chair Glide (CGS) in the chair with the label visible and the arrow pointing toward the back of the chair, Fig. 1.
2. Repositioning: Based on patient's weight have one or two caregivers on each side of the chair, grasp the handles and gently pull the system towards the back of the chair.
3. Each time the patient is raised off the chair, reposition the glide pad so that the red webbing is at the front of the chair.



Fig. 1

Easy to Clean

The Chair Glide pad only is impervious to fluids and can be wiped clean with soap and water or a mild disinfectant.

Caution

- DO NOT try to slide the resident towards the front of the chair since the one-way locking fabric is designed to only slide towards the back of the chair.
- Patient positioning should be done according to your facility's patient handling procedures
- Before each use check the system for signs of wear and replace when necessary