



Re-Order #	Description	Size	Unit
610220	Resident-Release Slide Control Wheelchair Belt	Universal	Each
610230	Resident-Release Slide Control Geri-Chair Belt	Universal	Each

Purpose

Indications:

The Skil-Care Cushion Slider Belt is recommended for use with a patient who:

- Has a tendency to slide in a wheelchair or geri-chair
- Requires a reminder not to leave the chair without assistance

Contra-indications:

Do not use this device for a patient who:

- Has a history of removing a quick release belt and attempts to leave the chair but is non-ambulatory
- Requires upper trunk control.

Wheelchair Application

1. Place the 2" nylon belt around the back of the wheelchair as in Figure 1.
Thread the belt between the armrest and side plate (Fig. 1).
2. Note that the pelvic belt can be applied in one of the three ways:
 - A. Under the wheelchair seat
 - B. Under the resident between the seat and the resident
 - C. Under a cushion.
 If placing as in "B" or "C", place the belt on the chair prior to seating the resident.
3. Once seated, thread the belt through the loop and then close the side release buckle (Fig. 2).
4. Throw the two straps under the chair ("A") or as in "B" or "C" through the space between the seat and back.
5. Wrap each strap around each hand bar at least once and attach the two straps together as in Fig. 3
6. Tighten the waist strap and then pelvic straps accordingly.

Fig. 1



Fig. 2



Fig. 3



Ger-Chair Application

1. Prior to seating the resident, place the 2" nylon belt around the geri-chair as in Figure 4.
2. Place the pelvic strap on the geri-chair seat (loop towards front) and thread the two straps through the seat and back (Fig. 5).
3. Once seated, thread the belt through the loop and then close the side release buckle (Fig. 6).
4. Attach the straps to the legs (Fig. 7), by opening the buckles and place around each leg accordingly. Pre-close the buckles and adjust the length of all straps as needed.

Fig. 4



Fig. 5



Fig. 6

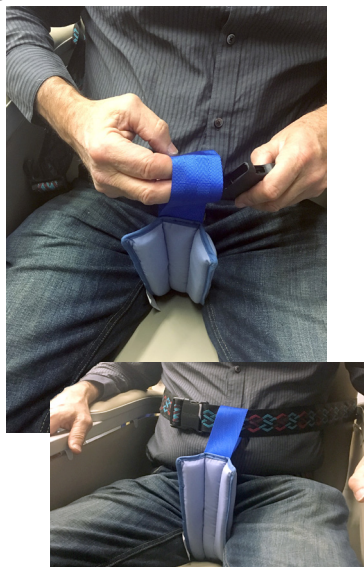


Fig. 7



Note: Geri-Chair application by threading through the seat is dependent upon the geri-chair design. If there isn't a gap, it may not be possible to apply the belt.

Easy Care

Belts can be laundered at temperatures below 180°F (82°C). Dry at the low setting. Do not use bleach directly on the fabric.

Caution

When the belt or buckles appear to be defective, the device should be replaced.

Visit us online at www.skil-care.com for our full line of nursing and therapy products.