



Re-Order #	Description	Size	Unit
610220	Slide Control Pad with 60" Belt	Universal	Each
610230	Slide Control Pad with 72" Belt	Universal	Each

Purpose

The Pelvic Slide Control Belt is designed to help minimize forward sliding in the wheelchair and geri-chair while promoting proper pelvic positioning. This device provides a gentle reminder cue for residents who attempt to exit the chair without assistance, while helping maintain safe posture and reducing fall risk.

OPERATION

- Place the pelvic belt on the chair as shown in figure 1.
- While feeding the two strap ends of the belt through and behind the chair, connect the straps together as shown in figure 2. Adjust the straps based on the patient size.
- Place 2" wide nylon gait belt around the back of the chair and position the open ends in front of the chair. See figure 3
- Thread the belt through the large loop of the slide control pad and connect the two ends of the plastic buckle, adjust to fit size of user. See fig 4

Note:

Wheelchair: The pelvic belt will extend slightly beyond the front edge of the chair, see figure 1.

Geri-Chair: Due to the depth on most chairs the belt will not extend pas the front edge.

Fig. 1



Fig. 2



Fig. 3



Fig. 4

