

29 Wells Avenue Yonkers, NY 10701 914-963-2040 800-431-2972

RESIDENT- RELEASE SLIDER BELT


Re-Order #

<u>Size</u> Universal

REALEZEE RELEASE SLIDER BELT

Re-Ord	er #	Ŀ
□6103	15	

<u>Size</u> Universal

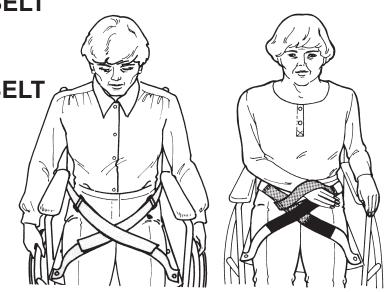

TO APPLY THE BELT

1. Before seating resident, lock both wheelchair brakes and rotate the footrests to the out-of-the-way position.

2. Seat resident as far back on the seat as possible.

Reposition the footrests and place patient's feet on them.

PURPOSE

The Resident-Release Slider Belt in intended to prevent residents from sliding off wheelchair seats.


CAUTION

These devices are not restraints! They are designed to be opened and removed by most patients. The determination as to the appropriateness of the Resident-Release Slider Belt for a specific patient is the responsibility of the facility and its professional and medical staff.


INSTALLATION TO THE WHEELCHAIR

1. Remove the front screws from the left and right sides of the wheelchair seat.

2. With the hook-and-loop material facing up, insert the screws through the grommets on the long belt sections. Reinsert the screws into the holes in the wheelchair seat. Tighten screws. (See Illustration A.)

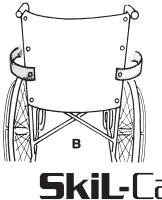

CORPORATION

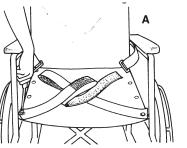


3. Behind the wheelchair, remove the screws second from the bottom on the left and right sides of the backrest. Insert the screws through the grommets on the **belt sections fitted with d-rings.** Reinsert the screws into the holes in the wheelchair backrest. Tighten screws. (See Illustration B.)

3. Thread the belt section from the resident's right side through the d-ring on resident's left side. Adjust the belt to a snug fit and press the hook-and-loop closure material together. Thread the belt section from the resident's left side through the d-ring on a the resident's right side. Adjust the belt to a snug fit and press the hook-andloop closure material together. (See Illustration C.) When properly applied the belt will be crisscrossed over the resident's lap as shown in the illustration at the top of this sheet.



CLEANING

The Resident-Release Slider Belt may be cleaned with a spray-type cleaner and a damp cloth or sponge. Although the belt can be laundered, this is not recommended because the hook-and-loop material tends to pick up laundry lint which reduces its holding strength.

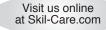

Advanced design for nursing and therapy.™