



Re-Order #	Description	Size	Unit
701050	Pelvic Slide Control Belt	Universal	Each

PURPOSE

The Pelvic Slide Control Belt is designed to help minimize forward sliding in the wheelchair and geri-chair while promoting proper pelvic positioning. This device provides a gentle reminder cue for residents who attempt to exit the chair without assistance, while helping maintain safe posture and reducing fall risk.

OPERATION

- Place the pelvic belt on the chair as shown in figure 1.
- While feeding the two strap ends of the belt through and behind the chair, connect the straps together as shown in figure 2. Adjust the straps based on the patient size.

Note:

Wheelchair: The pelvic belt will extend slightly beyond the front edge of the chair, see figure 1.

Geri-Chair: Due to the depth on most chairs the belt will not extend pas the front edge.

Fig. 1



Fig. 2

