

29 Wells Avenue Yonkers, New York 10701

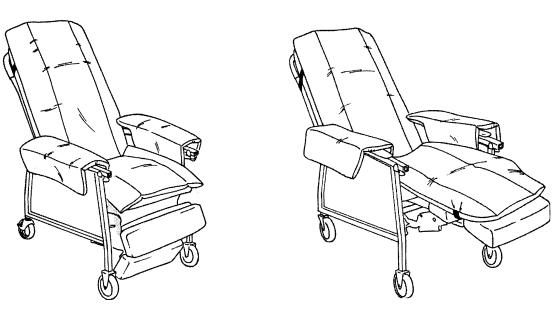
914-963-2040

# **GERI-CHAIR COZY SEAT**

Reorder # 703002 703003

### Description

Geri-Chair Cozy Seat/Backrest and seat Each Geri-Chair Cozy Seat/Backrest, Seat, leg-rest Each



Unit

#### **Purpose**

Skil-Care's Geri-Chair Cozy Seat provides comfort, pressure relief, and protection against skin tears. Designed to fit all styles of geri-chairs, the Cozy-Seat is reversible.

#### Instructions for Use

(Follow steps 1-3 for models 703002 and 703003.)

- 1. Place Cozy-Seat on geri-chair with label indicating "TOP" positioned at the top of the backrest.
- 2. Drape armrest cushions over armrests.
- 3. Take loose piece of hook and place it under the armrest in-line with the loop strap in Fig. 1. Then wrap the strap under the armrest and adhere to the hook as shown in Fig. 2.
- 4. Secure straps behind backrest/ (Illustration A)
- 5. (For model 703003) Secure elastic band around leg-rest. (Illustration B)

#### Laundering

The Geri-Chair Cozy Seat is completely launderable. Do not wash or dry at temperatures that exceed 180°F (82°C). Do not use bleach.

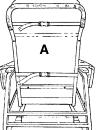


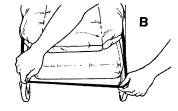



Fig. 2




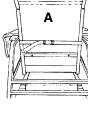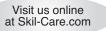Advanced design for nursing and therapy.™











29 Wells Avenue Yonkers, New York 10701


800-431-2972

# WHEELCHAIR COZY SEAT

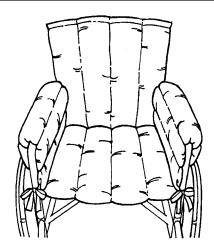
escription	Unit
Vheelchair Cozy Seat/ 16"	Each Each
	•

#### Purpose

Polyfoam padded and covered in brushed fabric. Skil-Care's Wheelchair Cozy Seat provided comfort and warmth. It protects against contact with metal wheelchair parts and cushions the armrests.



914-963-2040

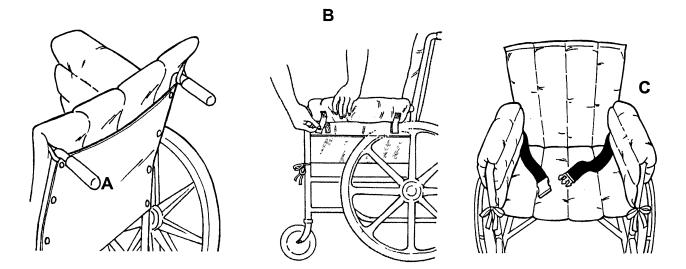

## Easy to Use

- 1. Place the Cozy-Seat on the wheelchair and loop the elastic backrest bands over the handles. See illustration A.
- 2. Secure the Velcro<sup>R</sup> closures on the left and right sides of the armrests. Secure the ties on the left and right sides. See illustration B.
- 3. If the wheelchair user requires a safety or positioning belt, the belt can be brought through the openings at the lower left and right sides of the backrest. See illustration C.

#### Easy Care

The Cozy Seat is machine launderable. Do not wash or dry at temperatures that exceed 180°F (82°C). Do not use bleach.

