

Re-Order#

705031

705032



Purpose

This stably mounted tray flips out of the way for easy exiting. The side and armrest is padded (see, Fig. 1) when the tray is in the down or flipped position. The tray provides two cup holders that can be used for beverages or small personal items such as pencils, pens or eye glasses

Easy to Use

(Instructions are the same for right-arm or left-arm devices).

1. Remove left or right wheelchair armrest pad. (Illustration A)

Description

- 2. Place slotted base on armrest support bar so that slots face up and notch fits around backrest support bar. (Illustration B)
- 3. Place the large washers on the long bolts. Position the base so that the screw holes in the armrest support bar are visible through two of the three slots. Insert the long bolts through these slots and through the holes in the armrest support bar. (Illustration C) Place a small washer on the end of each bolt and secure with a wing nut. (Illustration D)
- 4. Wrap the Velcro® strap indicated for "Full-Arm Attachment" (see label on support base) around vertical bar at front of chair and insert through slot on opposite side of base. (Illustration E-1) Press arm support base against bar while wrapping Velcro® strap as tightly as possible to stabilize base. Secure Velcro® strap. (Illustration E-2) The wheelchair tray is now installed. Reattach the foam pad (see, Fig. 1)

Easy Care

This durable, scratch resistant plastic can be cleaned with soap and water or with disinfectant. The removable arm pad can be wiped clean with a standard cleanser. If disinfected, avoid using chlorine (bleach) chemicals.

