


Universal Storage Bag

Reorder #	Description	Size	Wt.	Unit
□707010	Blk vinyl (3 pocket)	11.5"W x 1"D x 12"H	.06 lbs	Ea

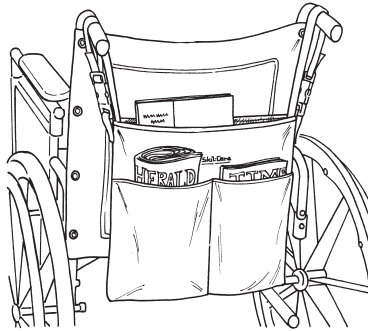

PURPOSE

When an individual is using a wheelchair, walker or other aid, it is convenient to locate personal items in a storage bag. The universal bag is sufficiently large to house most personal items and can be located on the wheelchair, walker or other similar type uses.

USE INSTRUCTIONS

The storage bag can be placed on the two wheelchair handles using the loops. If necessary, the loops can be adjusted in length and the strap may be opened with the two side release buckles. The side release buckles can be opened by simultaneously pinching the side buckle snaps.

CARE INSTRUCTIONS

The bags can be clean or laundered while maintaining wash and dry temperatures below 180°F (82°C). The vinyl bag can be wiped with a mild detergent.