

Reorder#	Unit
707030	Each





## **Purpose**

Skil-Care's Modesty Apron provides dignity and convenience for female residents in wheelchairs or geri-chairs. This full-sized apron is an appreciated modesty cover that extends well past the knees. It features a large pocket that provides easy access to personal items

## Easy to Use

Place the apron at the resident's waist and secure the slidelook buckle. Notice: This apron is neither designed nor intended to be used as a physical restraint. Do not secure strap around the back of the chair. Doing so will impede the resident's autonomy.

## **Easy Care**

The Modesty Apron may be laundered and dried at temperatures up to 180°F (80°C). Do not use bleach.

