Available Low-Shear cover

easy care.

Vinyl cover is incontinent-proof and _____wipes clean for easy care.

reduces skin friction and shear. Fluid-proof. Wipes clean for

| Re-Order # | Description | Size | Unit |
|------------|---|------------------|------|
| 751026 | Gel Foam X-Cushion - Vinyl Only | 18" x 16" x 2.5" | Each |
| 751028 | Gel Foam X-Cushion - Vinyl Only | 18" x 18" x 2.5" | Each |
| 751027 | Gel Foam X-Cushion - Vinyl w/Low Shear I Cover | 18" x 16" x 2.5" | Each |
| 751029 | Gel Foam X-Cushion - Vinyl w/Low Shear I Cover | 18" x 18" x 2.5" | Each |
| 751037 | Gel Foam X-Cushion - Vinyl w/Low Shear I Cover | 20" x 18" x 2.5" | Each |
| 751030 | Gel Foam X-Cushion - Vinyl Only | 16" x 16" x 2.5" | Each |
| 751031 | Gel Foam X-Cushion - Vinyl w/Low Shear I Cover | 16" x 16" x 2.5" | Each |
| 751032 | Gel Foam X-Cushion - Vinyl Only | 20" x 16" x 2.5" | Each |
| 751033 | Gel Foam X-Cushion - Vinyl w/Low Shear I Cover | 20" x 16" x 2.5" | Each |
| 751034 | Gel Foam X-Cushion w/Coccyx Cut-Out and Low Shear I Cover | 16" x 16" x 2.5" | Each |
| 751035 | Gel Foam X-Cushion w/Coccyx Cut-Out and Low Shear I Cover | 18" x 16" x 2.5" | Each |
| 751036 | Gel Foam X-Cushion w/Coccyx Cut-Out and Low Shear I Cover | 20" x 16" x 2.5" | Each |

____ Straps with buckle secure cushion to wheelchair.

ŠkiĽ-Care

Water-based gel surface distributes weight to relieve pressure. Its cooling action reduces skin damaging perspiration

New X-Design creates four separate gel sections for maximum stability.

Purpose

Skil-Care's Gel Foam X-Cushion is a combination gel and foam cushion designed to relieve pressure. The waterbased gel surface provides cooling action that reduces skin-damaging perspiration.


Important Safety Reminder

The Gel Foam X-Cushion with a cloth outer cover has straps with buckles that must be secured to the wheelchair to prevent the cushion from sliding off the seat.

Easy Care

The vinyl cover may be wiped clean using a damp cloth and spray-type cleanser. A mild disinfectant may be used. The Low-Shear outer cover can be laundered and dried at temperatures that do not exceed 180°F (82°C). Do not use bleach.

This product is guaranteed for 6 months.

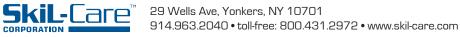

High-density foam base prevents

bottoming-out and increases sitting comfort.