



Re-Order #	Description	Size	Unit
753400	EZ Dry Foam Cushion w/LSII Cover	16"W x 16"D x 2"H	Each
753402	EZ Dry Foam Cushion w/LSII Cover	18"W x 16"D x 2"H	Each
753404	EZ Dry Foam Cushion w/LSII Cover	20"W x 16"D x 2"H	Each
753406	EZ Dry Foam Cushion w/LSII Cover	16"W x 16"D x 3"H	Each
753408	EZ Dry Foam Cushion w/LSII Cover	18"W x 16"D x 3"H	Each
753410	EZ Dry Foam Cushion w/LSII Cover	20"W x 16"D x 3"H	Each

Purpose

Skil-Care offers a complete listing of cushions that provide pressure relief and redistribution. For incontinent residents heat sealed vinyl covers are available that prevent the migration of body fluids into the cushion. Several cushions are also offered with sewn covers that are water proof but in some situations fluids migrate into the foam through the stitching. Although standard polyester foam can be laundered, it takes several hours to rack dry. When foam cushions are preferred, especially for incontinent residents, the EZ Dry foam cushions have many favorable characteristics.

Cushion Attributes

Reticulated foam cushions have large pores that allow the cushion to dry easily and quickly. Foam pore construction whisks away moisture and is cooler. These cushions can also be used outside in inclement weather. Soft foam provides maximum pressure relief and redistribution. The low shear II outer cover has a maximum stretch for minimizing shear forces and is wipe clean with tie straps.

Easy to Use

Place the cushion on the chair with the non-slip covering on the bottom and the tie straps towards the back. Affix the straps to the back of the wheelchair and tie securely. If replacement covers are needed, refer to the universal cover listing in the Skil-Care product catalog.

Easy Care

The durable LSII cover wipes clean with soap and water or a mild disinfectant. If the foam needs to be washed, it should be hand washed and rack dried.