


Lateral Positioning Cushion

Reorder #	Size
756010	18" x 16" x 3½" x 5"

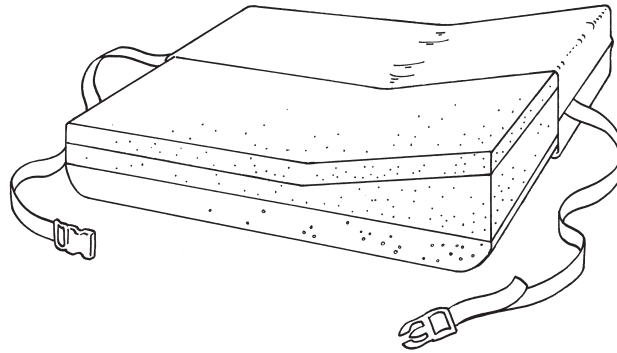

Purpose

Skil-Care Lateral Positioning Cushion controls lateral leaning while reducing sitting pressure and eliminating wheelchair sling-seat hammocking.

Effective Design

The elevated side of this cushion controls lateral leaning. A top layer of memory foam conforms to the user to reduce pressure-causing high and low spots, while the middle layer of high-density foam provides stability and prevents bottoming out. The firm contoured base distributes user's weight and eliminates sling-seat hammocking. A liquid-proof cover features a low-shear top to protect fragile skin against damage caused by friction and shear. The slip-resistant bottom keeps the cushion in place.

Easy to Use

Place the Lateral Positioning Cushion on the wheelchair seat so that the elevated side of the cushion is properly located to address the user's lateral leaning problem. The elevated side may be placed on either the left or right side of the wheelchair seat.

Important Safety Feature

The Lateral Positioning Cushion has a safety strap that must always be secured to the wheelchair to prevent the cushion from slipping off the seat.

Easy Care

The cover may be wiped clean with spray-type cleanser and disinfected with a non-chlorine disinfectant. Do not launder.