


# Wheelchair Leveling Pad

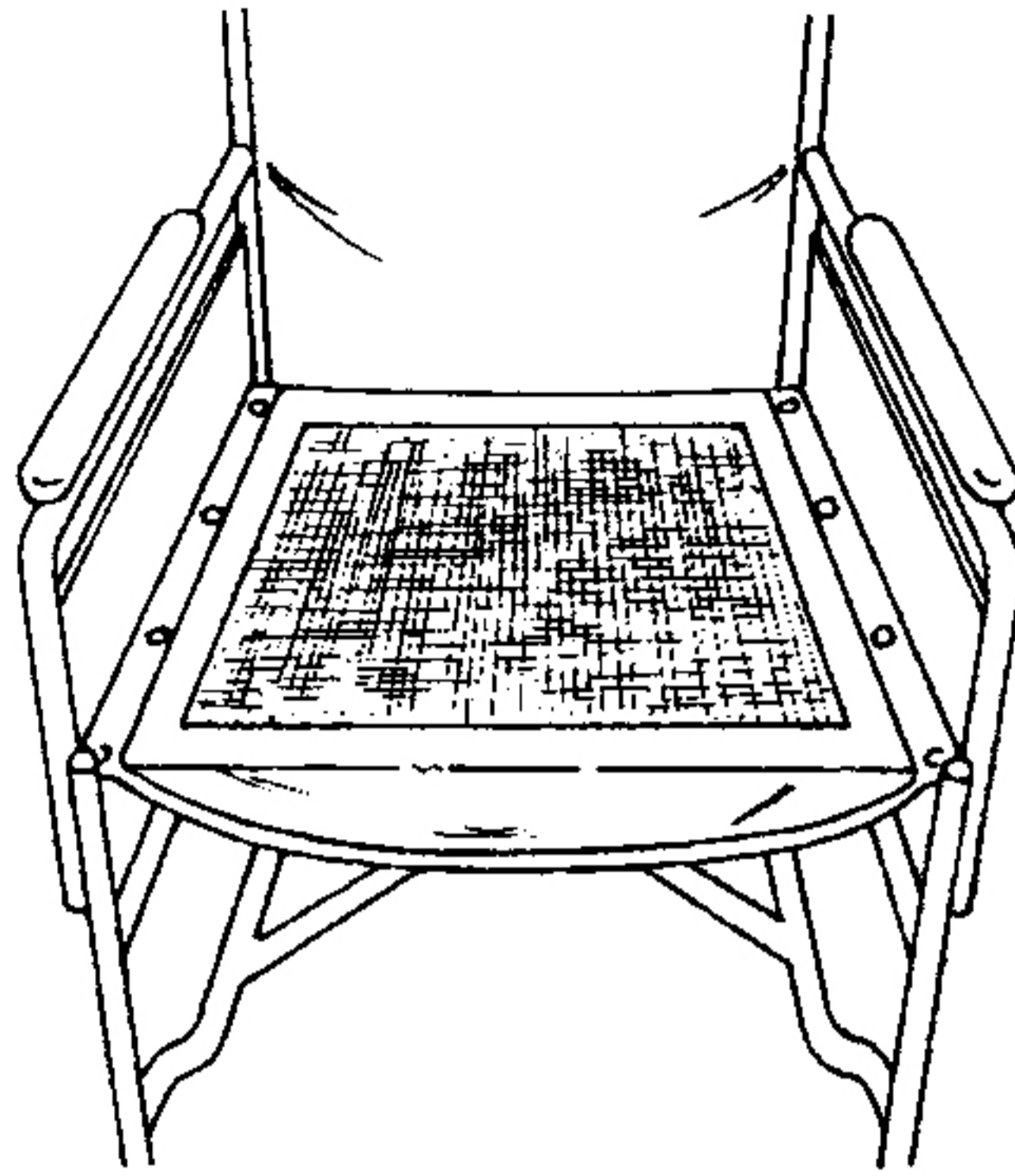

| Reorder#                        | Description                         | Size      | Unit |
|---------------------------------|-------------------------------------|-----------|------|
| <input type="checkbox"/> 757010 | Wheelchair Leveling Pad/No Cover    | 18" x 16" | Each |
| <input type="checkbox"/> 757015 | Wheelchair Leveling Pad/Vinyl Cover | 18" x 16" | Each |
| <input type="checkbox"/> 757017 | Wheelchair Leveling Pad/Vinyl Cover | 20" x 16" | Each |

## Purpose

This solid, closed-cell foam insert eliminates wheelchair sling-seat hammocking. This reduces pressure on the coccyx and spine, lowers interface pressure on the buttocks, and controls the inner-thigh that contributes to poor sitting posture and sliding. The Wheelchair Leveling Pad provides a perfect foundation for all therapeutic cushions.

## Important Notes

1. The leveling pad is to be used only with a cushion. Do not seat wheelchair user directly on the pad.
2. The leveling pad is packaged with Skil-Care's Slip-Grip anti-slide material. This must be placed on the pad to prevent the cushion from sliding off. See illustration above. If cushion is provided with tie straps, secure these to the wheelchair frame.

## Easy Care

The Wheelchair Leveling Pad **without cover** may be wiped using a damp cloth only. The leveling pad **with vinyl cover** may be cleaned using a damp cloth and spray-type cleanser. A mild disinfectant may also be used on the vinyl.