

Re-Order #	Description	Type	Length	Unit
909370	Metal Press Release Seat Belt Alarm with ChairPro Alarm Unit and Grommets	Wheelchair	38"L - 50"L	Each
909373	Metal Press Release Seat Belt Alarm with ChairPro Alarm Unit and Grommets	Wheelchair	50"L - 70"L	Each
909385	Metal Press Release Seat Belt Alarm with ChairPro Alarm Unit and Adjustable Loop Attachment	Wheelchair	45"L - 55"L	Each
909386	Metal Press Release Seat Belt Alarm with ChairPro Alarm Unit and Adjustable Loop Attachment	Wheelchair	50"L - 65"L	Each
909392	Metal Press Release Seat Belt Alarm with Grommets Only (ChairPro/MultiPro/BedPro Alarm sold separately)	Wheelchair	38"L - 50"L	Each
909394	Metal Press Release Seat Belt Alarm with Grommets Only (ChairPro/MultiPro/BedPro Alarm sold separately)	Wheelchair	50"L - 70"L	Each
914117	Metal Press Release Seat Belt Alarm with Adjustable Loop Attachment Only (ChairPro/MultiPro/BedPro Alarm sold separately)	Geri-Chair	45"L - 55"L	Each
914118	Metal Press Release Seat Belt Alarm with Adjustable Loop Attachment Only (ChairPro/MultiPro/BedPro Alarm sold separately)	Geri-Chair	50"L - 65"L	Each

*Note: Please see page two to calculate appropriate belt size*

Skil-Care’s new ChairPro Seat Belt Alarm combines a wheelchair belt with an alarm for double wheelchair safety. The belt attaches to the chair and provides security while the user is seated. When the buckle is opened, an alarm is activated. To assist long-term acre facilities maintain a restraint-free environment, Skil-Care has selected a durable buckle that is easily released by most wheelchair users.


PURPOSE

The ChairPro Seat Belt Alarm is designed to alert caregivers when a patient/ resident has released the wheelchair safety belt buckle and may leave from the wheelchair.

CAUTIONS

1. The ChairPro Seat Belt Alarm is not a substitute for proper supervision. Fall-risk and wander-risk patients or residents should be monitored regularly.
2. Check battery each time alarm is used. If weak, replace the 9-volt battery.
3. LED should glow when buckle is closed. If LED fails to glow, check battery. Replace if necessary.


INSTALLATION  
Alarm Unit

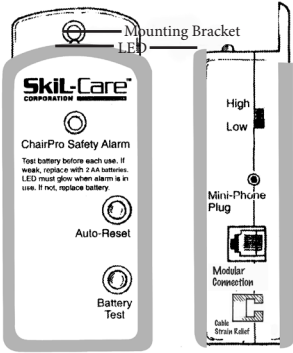



You may choose one of the following two ways to attach the alarm unit to the wheelchair.

Attachment of Alarm Unit  
Mounting Bracket

1. Remove the second-from-top screw from either the left or right side of the wheelchair backrest.
2. Remove grommet from screw, place the screw through hole on alarm unit mounting bracket, and fasten screw to wheelchair backrest. (Illustration A)

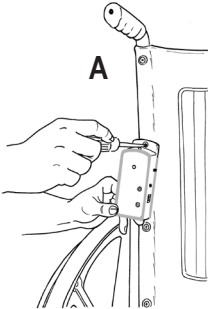

Cord Loop

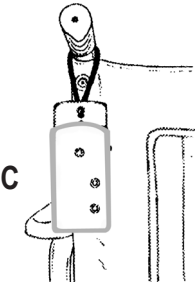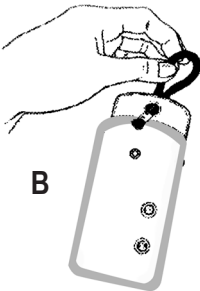



1. Insert enclosed cord loop through bracket hole. (Illustration B)
2. Hang from wheelchair push-handle (Illustration C)



SEAT BELT INSTALLATION

WHEELCHAIR BELT INSTALLATION (Grommet)

1. Remove the rearmost screws from the left and right sides of the wheelchair seat. (Illustration D)
2. Remove the grommets from the screws, place screws through the grommets at the ends of both belt sections, and refasten the screws to the wheelchair seat. (Illustration E)
3. Select either hole in belt depending upon desired length, closer holes will yield a shorter length, holes closet to the end will yield a longer length.
4. Place connector cord through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

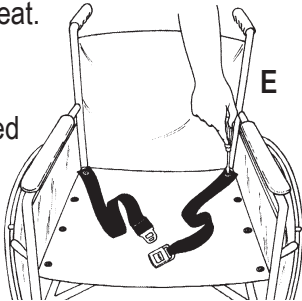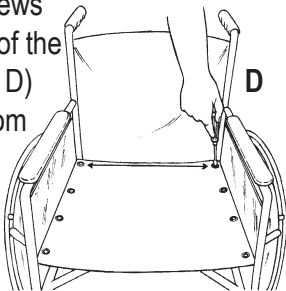

WHEELCHAIR BELT INSTALLATION (Loop)

1. Unthread the loop end of the belt.
2. Lay belt on wheelchair while holding loop end, place loop end around the bottom of the wheelchair (Fig. 1).
3. Place belt end through the loop and pull slowly (Fig. 2)
4. Pull end until taut. (Fig. 3)
5. Insert both ends of the belt through the opening in the chair and place the belt around the resident and adjust so that it is snug but not uncomfortable.



Fig. 1



Fig. 2



Fig. 3

GERI-CHAIR/UTILITY CHAIR INSTALLATION (Loop)

- 1. Push white loops through back upholstery where seat and back meet. (Fig. A)
- 2. On back side of recliner, take each white loop with plastic buckle and attach around the bar closest to where the belt came through the upholstery. (Fig. B)
- 3. Adjust white loop so you can visibly see the black belt ends. (Fig. C)
- 4. With patient seated, adjust slide buckle to a comfortable fit around the waist.
- 5. Attach alarm to chair and connect belt to the alarm.

Note: There are adjustments with both the slide buckle on the black belt and the white loops.

Geri-Chairs/Utility Recliner Chairs have many various designs. Caregiver may have to use an attachment method that is different as described.



Fig. A

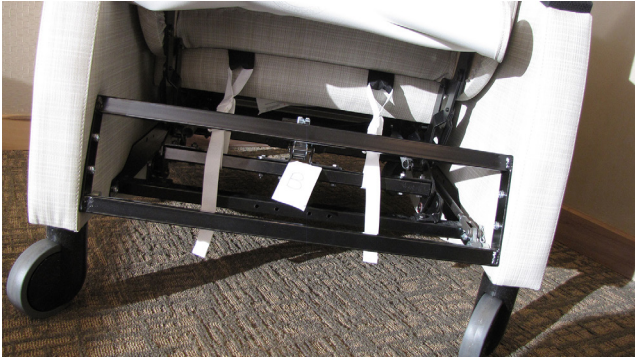


Fig. B


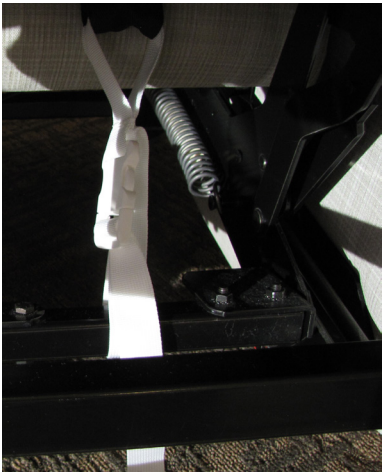



Fig. C

OPERATION

- 1. Remove battery compartment cover on left side of alarm unit and install two “AA” batteries.
- 2. Press Battery Test button. If alarm doesn’t sound, check battery connection and retest. If alarm fails to sound, replace battery.
- 3. Insert mini-phone jack into Mini-Phone Plug receptacle on right side of alarm. (Illustration G) Alarm will sound. Press Auto-Reset to silence alarm.
- 4. Before using, test unit by inserting the metal tab into the buckle. Check alarm to make certain that LED is glowing. Then release buckle. Alarm should sound. Press Reset button to silence alarm. If alarm fails to sound, make certain that spade connectors and phone jack are securely in place.
- 5. Select High or Low alarm volume using switch on right of alarm.
- 6. **Important: If the alarm is activated, it may be silenced by pressing the Auto-Reset button. The alarm is now in stand-by mode. Do not secure buckle when unit is not being used. Doing so will cause the LED to glow and shorten battery life. Alarm will automatically reset itself when the metal tab is inserted into the buckle. There is no On/Off switch.**
- 7. Place belt around user’s waist and secure by placing tab into buckle. Adjust belt to proper fit. Use the slider buckle to adjust the length of belt. If the belt is too long or short see step 4 under “Wheelchair Installation (Grommet)”.
- 8. The belt is released by pressing the “Press” button. (Illustration H)

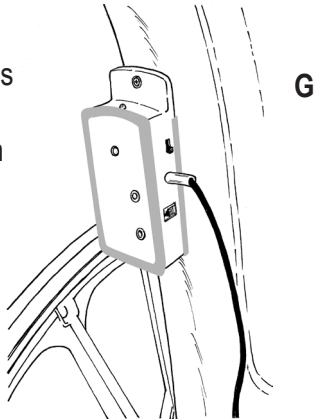

G

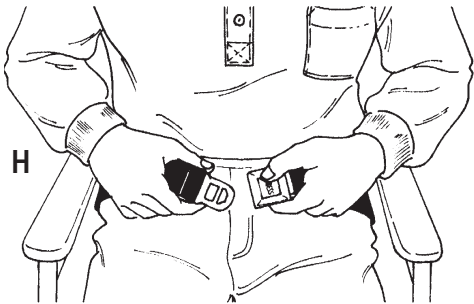

H

SAFETY REMINDERS


- 1. Check battery before each use. If alarm is weak or fails to sound, replace the two “AA” batteries.
- 2. When unit is in use, check LED regularly. If it is not glowing, replace the two “AA” batteries.

GUARANTEE

The alarm unit, belt, and buckle are replacement guaranteed against defects in materials and workmanship under conditions of normal use for a period of one year from date of purchase. Please return defective part to dealer from whom the system was purchased. The dealer will send the defective part to Skil-Care and arrange for shipment of a replacement. The guarantee does not cover the batteries supplied with the unit.

How to determine the appropriate belt size

Take width ‘W’ of the Geri-Chair/Recliner/Wheelchair and add that to the length of ‘B’ and compare this total number to the residents belt size. (see diagram)

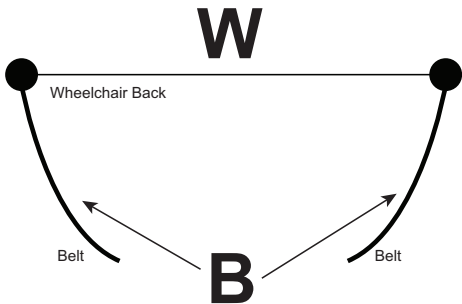

Distributed by: Skil-Care Corporation



29 Wells Ave, Yonkers, NY 10701  
914.963.2040 • toll-free: 800.431.2972 • www.skil-care.com

Rev: 051721