





Re-Order#	Description	Units
909376	Gel-Foam Toilet Seat Cushion Alarm System* (ChairPro Unit)	Set
909377	Gel-Foam Toilet Seat Cushion Alarm System*(MultiPro Unit)	Set
909397	Replacement Straps	6/pk of 4
909367	ChairPro Alarm Unit Only	Ea.
909510	MultiPro Alarm Unit Only	Ea.
909378	Gel-Foam Toilet Seat Replacement Cushin w/Sensor	Set
909398	Universal Wall Bracket	Ea.

*One Year Guarantee

Fig. 1

Purpose

The Gel-Foam Toilet Seat Cushion Alarm System incorporates a top layer of gel and a bottom layer of foam to provide comfort and maximum pressure relief. The baffled gel pack design stabilizes the gel to insure that the gel doesn't migrate away from the high pressure areas. The heat-sealed vinyl casing is water proof and cleans easily with disinfectants and is soft and gentle to the skin.

Easy to Use

Installation of Toilet Seat Sensor:

Make sure that the cable is facing the back, away from the split. The toilet seat cushion should be placed over the toilet seat with the gel surface facing up. The toilet seat cushion is designed to be placed on most toilet seats and is held in place by four removable straps as shown in (Fig. 1). Each strap is to be threaded through both rectangular rings (Fig. 2) and are closed by placing the end of the strap on top of the opposing side. The strap should be pulled snug but not too tight. All toilet seat cushion alarms have a sensor imbedded in the cushion and are guaranteed for one year. Model 909376 includes the ChairPro electronic unit in addition to the cushion and imbedded sensor. Model 909377 substitutes the ChairPro alarm with the MultiPro alarm. The MultiPro alarm system should be used when in addition to an audible alarm, the caregiver is interested in a nurses call interface. Please refer to the Skil-Care catalog or website www.skil-care.com for specifics on the ChairPro and MultiPro electronic units.









Fig. 2

Easy to Use (cont.)

After the cushion is affixed to the seat, locate the alarm as convenient with or without the wall bracket. Connect the grey wire from the pad to the modular receptacle in either the Chair or MultiPro unit. Be sure that the wire will not interfere with the raising and lowering of the seat, or doesn't impair the residents movement or isn't a choking hazard.

Test the alarm by sitting on the gel-foam toilet seat cushion and rising up off the seat. If the alarm fails to sound check the battery and/or cable connection.

Note: The imbedded sensor is only located in one quadrant so unless the sensor is activated by sitting on the entire cushion, it may not activate.

Easy Care

The vinyl seat cushion is heat-sealed and can be cleaned with a mild disinfectant. Each strap can be removed and cleaned by wiping, spraying or soaking in a disinfectant. Do not boil or place in a microwave. If the straps require replacing they can be ordered as (model# 909397). If a tear or rip is observed, the unit should be discarded.

Caution

Although the gel is non-toxic, medical care should be consulted if the gel is ingested. This product is an aid and shouldn't be a substitute for adequate medical attention.

Warning

- 1. Check alarm before each use.
- 2. Do not immerse alarm cushion in water
- 3. This device will alert caregiver if a resident stands up. It is not a restraint and isn't intended to deter a resident from standing up.
- 4. This device isn't a substitute for proper caregiver supervision.
- 5. Although the imbedded sensor may operate for longer than one year, it is only guaranteed for one year.

