



Re-Order #	Description	Size	Unit
909388	BioShield Alarm Belt with Adjustable Loop Attachment	Universal	Each

PURPOSE

The BioShield Alarm Belt with Adjustable Loop Attachments is a **safety mobility device** offering a non-restrictive wheelchair belt featuring a buckle-activated alarm sensor. It is designed to alert caregivers prior to a resident leaving a wheelchair or chair, reducing the risk of falls or unsupervised transfers. When fastened, the belt places the monitor in “active” mode. If the buckle is released, the alarm will sound, notifying the caregiver. Once re-secured, the system resets automatically and returns to monitoring mode. This version features a BioShield infection-control polyurethane coating that resists fluid penetration and simplifies cleaning and disinfection, enhancing hygiene in multi-user or clinical environments.

OPERATION

- The BioShield Alarm Belt is not a substitute for direct supervision. Residents at fall or wander risk must be regularly monitored.
- Check the monitor’s battery before each use. A flashing green light indicates monitoring mode.
- Test before every use: Fasten the buckle (green light flashes), then unbuckle. The alarm should sound.
- Press the Reset button on the alarm to silence it.
- If the alarm fails to activate, verify proper cable connection and function. If the issue persists after checking the battery and cord, discontinue use and contact Skil-Care for service.

GUARANTEE

The BioShield Alarm Belt is guaranteed for one year from the date of purchase against defects in materials and workmanship under normal use. Return defective parts to the dealer from whom purchased. The dealer will coordinate replacement with Skil-Care.

LOOP ATTACHMENT

1. Feed each end of the belt loops around the sides of the wheelchair back so that both straps hang down the rear of the chair.
2. Before securing, cross the two adjustable black webbing loops (crisscross them) behind the wheelchair for added stability.
3. Route the crisscrossed loops around the wheelchair kick spurs (rear lower frame bars).
4. Adjust each loop to the desired length to ensure a secure and snug fit.
5. With the resident seated, bring the belt across the lap and fasten the buckle at the front, adjusting for comfort and proper tension.
6. Attach the alarm unit to the back of the chair and plug the belt connector into the alarm.
7. Confirm that all cords and straps are free of moving parts, wheels, and pinch points before use.

