


# Clip-On Alarm Mounting Bracket And Universal Wall Bracket

Reorder #	Unit
<input type="checkbox"/> 909390	Each
<input type="checkbox"/> 909398	Each

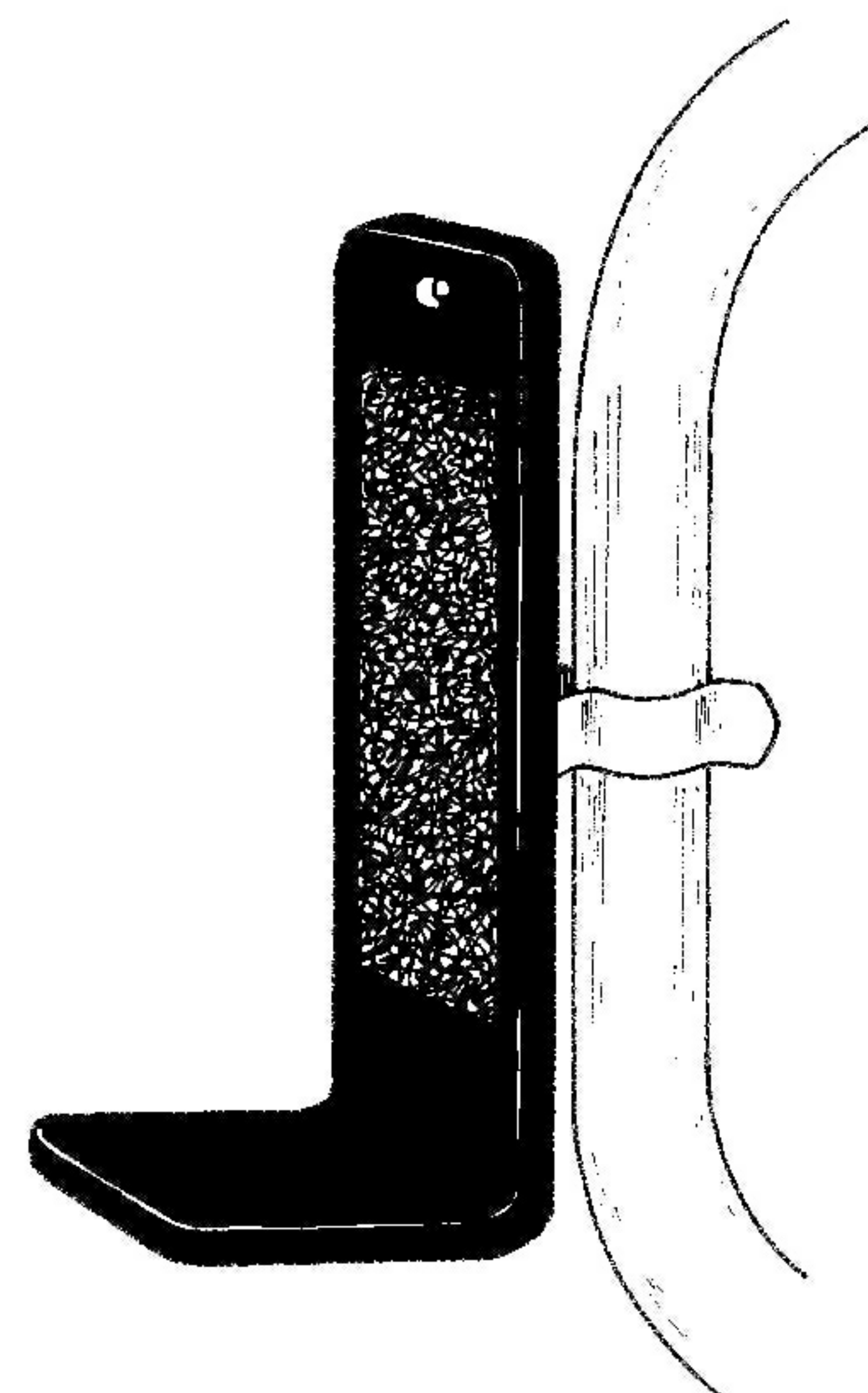

## Purpose

Skil-Care's Clip-On Alarm Mounting Bracket is designed to permit caregivers to attach wheelchair alarm units to any part of the wheelchair frame, safely out of the resident's easy reach. It also allows an alarm unit to be moved easily from one chair to another or to be removed when the chair is power washed.


## Instructions for Use

1. The Clip-On Alarm Mounting Bracket includes a strip of self-adhesive Velcro<sup>®</sup> hook material. Peel the backing off the Velcro<sup>®</sup> strip and attach it to the back of the alarm unit. The Velcro<sup>®</sup> may be cut and sized to avoid covering a battery compartment cover on the back of the alarm unit.
2. Rest the alarm unit on the base of the bracket and press it against the loop material on the back of the bracket to secure the unit.
3. Clip the bracket to the any part of the tubular wheelchair frame. (See illustration)

## Alternate installation

The bracket may be permanently installed to the wheelchair. This is accomplished as follows:

1. Remove the two screws that secure the clip to the bracket.
2. Remove one of the screws that secure the backrest to the wheelchair frame.
3. Place this screw through the hole at the top of the bracket and reattach the screw to the wheelchair.

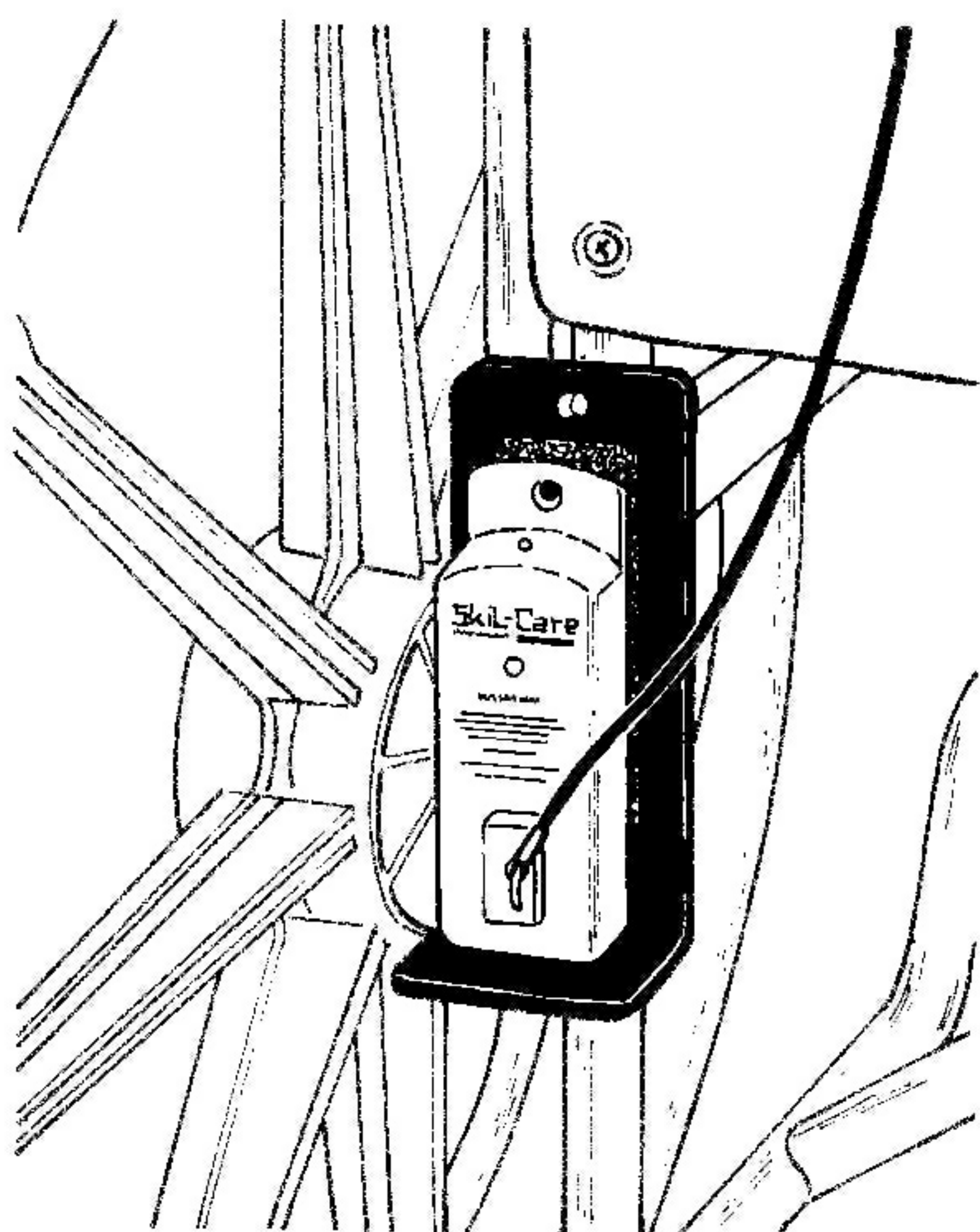

Model number 909390 is  
only for the wheelchair



Model number 909398 is  
only used for the wall