




## ChairPro™ / MultiPro™ UNDERSEAT ALARM


U.S. Patent No. 6,963,286

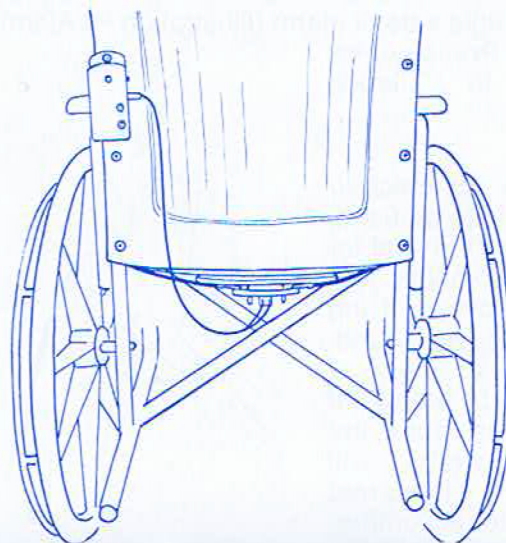


Model #	Fits Wheelchairs	Unit
909391	18" - 22"	Set

### INSTALLATION-UNDERSEAT SENSOR UNIT


1. The under-seat sensor unit includes a quick-detach feature that permits the sensor to be removed easily when the wheelchair has to be power washed or when wheelchair is folded. (Illustration A)

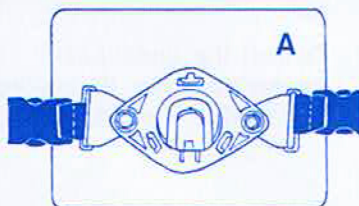

2. Remove the second and third screws on both sides of the wheelchair seat. (Illustration B)

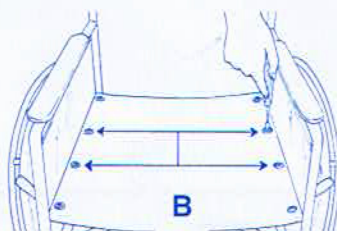


3. Disconnect the hanger straps from the sensor unit. Place the web straps with grommets between the wheelchair seat upholstery and the seat frame. Align the grommets with the screw holes. Reinsert and fully tighten the screws. The hanger straps should now be secured as shown. (Illustration C)

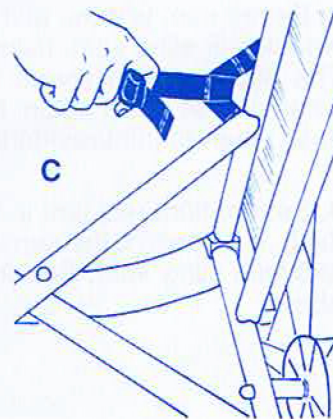


### UNDERSEAT SENSOR UNIT


4. Connect both hanger straps to the sensor unit. (Illustration D) Make certain that the prongs on the sensor's plug are facing toward the back of the seat.

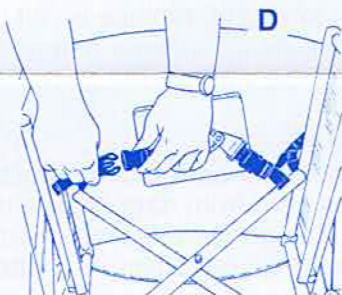

5. Pull the loose ends of both hanger straps to center the sensor and bring it into contact with the underside of the wheelchair seat. (Illustration E)

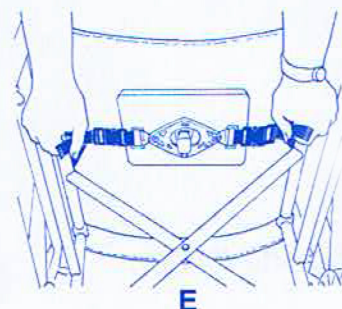

6. When installation is completed, the under-seat sensor, quick-detach feature, and power cord should appear as shown. (Illustration G)

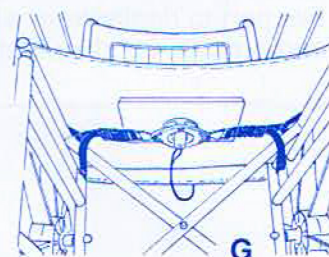

7. If you plan to use a wheelchair cushion, see #4 under "Operation" below.

Continued on reverse side.