



Re-Order #	Description	Belt Length	Unit
909392	Metal Press Release Seat Belt w/Grommets	50"L	Ea.
909394	Metal Press Release Seat Belt w/Grommets	70"L	Ea.
914117	Metal Press Release Seat Belt w/Adjustable Loop Attachment	45"L	Ea.
914118	Metal Press Release Seat Belt w/Adjustable Loop Attachment	65"L	Ea.

PURPOSE

The Seat Belt with Buckle Sensor is a **safety mobility device** offering a non-restrictive wheelchair belt featuring an integral buckle-activated alarm switch. It is designed to alert caregivers before a resident leaves the chair, reducing the risk of falls or unsupervised transfers. When the belt is secured around the wheelchair and the buckle is fastened, the system is in “monitoring” mode. If the resident attempts to get up or exit the wheelchair by releasing the buckle, the alarm activates, alerting the caregiver. Once the buckle is refastened, the system automatically resets and returns to monitoring mode.

OPERATION


- The Seat Belt with Buckle Sensor is not a substitute for proper supervision. Residents who are fall-risk or wander-risk should be monitored regularly.
- Check the battery each time the monitor and belt are used. The monitor’s main light should flash green when the buckle is fastened.
- Before each use, test the unit by fastening the buckle. Once the monitor’s main light begins blinking green, unbuckle the belt, the alarm should sound.
- Press the Reset button to silence the alarm.
- If the monitor fails to alarm, check the sensor connection and internal switches. If the issue persists after installing new batteries and confirming the belt is properly connected, discontinue use and contact your Skil-Care representative.

GUARANTEE

The Seat Belt with Buckle Sensor is guaranteed for one year from the date of purchase. This guarantee covers defects in materials and workmanship under normal use. To obtain service, return the defective part to the dealer from whom it was purchased. The dealer will forward the part to Skil-Care and arrange shipment of a replacement.

GROMMET INSTALLATION

1. Remove the rearmost screws from the left and right sides of the wheelchair seat. (Illustration D)
2. Place screws through the grommets at the ends of the belt. Select either hole in the belt depending upon desired length, closer holes will yield a shorter length, holes closet to the end will yield a longer length. Then refasten the screws to the wheelchair seat. (Illustration E)
3. Place mini phone jack cord through the space between the wheelchair seat and backrest. Make certain that it is away from wheels and other moving parts.

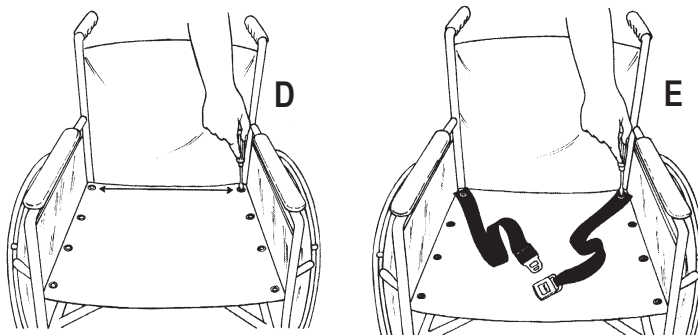

LOOP INSTALLATION

1. Push white loops through back of chair on each side so both white loops are hanging out the backside of the chair. (Fig. A).
2. On back side of chair, take each white loop with plastic buckle and attach around the metal bar (Fig. B)
3. With patient seated, adjust belt to waist size of patient by reducing or expanding white loop straps.
4. Attach alarm to chair and connect belt to the alarm.
5. Geri-Chairs/Utility Recliner Chairs and Wheelchairs have many various designs. Caregiver may have to use an attachment method that is different as described.



Fig. A



Fig. B