



Re-Order #	Description	Size	Units
909530	Gel-Foam Cushion Alarm Sensor	16" x 16" x 2"	Each
909531		18" x 16" x 2"	Each
909532		20" x 16" x 2"	Each

### Purpose

This Gel-Foam Cushion with built in sensor provides pressure relief, redistribution and comfort.

### Easy To Use

Place the Gel Foam Cushion on the wheelchair, positioning the safety straps and alarm cable toward the back. Secure the straps and guide the alarm cable through the space between the wheelchair seat and backrest. Make sure the cable is kept clear of wheels and any moving parts. Once the cushion is plugged into the monitor, the alarm will activate when the user rises and automatically reset when he or she sits back down.

### Easy Care

The Gel Foam Cushion Alarm Sensor is protected with a Low-Shear I launderable cover for easy maintenance. Do not wash or dry the cover at temperatures over 180°F (82°C), and bleaching is not recommended. The cushion itself is sealed in durable vinyl, which can be wiped clean with mild soap, water, or a non-bleach disinfectant.