



Re-Order #	Description	Size	Units
909535	Leveling Pad Alarm Sensor	Fits 16"-18' WC	Each
909389	Replacement Alarm Sensor for Leveling Pad and Gel-Foam Cushion	Universal	Each

Purpose

The Skil-Care Leveling Pad Alarm Sensor helps reduce fall risks by providing a stable surface and triggering an alert when a resident attempts to leave their seat. It combines safety monitoring with added comfort and support.

Easy To Use

The Skil-Care Leveling Pad Alarm Sensor is simple to use. Place the sensor flat and centered on the wheelchair seat, then place a cushion on top of the pad before the resident sits down. Connect the sensor cord securely to the alarm unit, and position the resident so their weight activates the sensor. When the resident begins to rise or shift off the pad, the alarm will sound to alert caregivers. To reset, simply reposition the resident on the pad.

Easy Care

The Skil-Care Leveling Pad Alarm Sensor is covered in durable black vinyl that can be wiped clean with mild soap and water. A mild disinfectant may also be used, as long as it does not contain bleach or harsh chemicals.