

BARIATRIC FOAM CUSHION

<u>Re-Order #</u>	<u>Description</u>	<u>Reorder#</u>	<u>Size</u>	<u>Cover</u>
□910140	20" x 18" x 4.5" x 3"H	□910150	24" x 18" x 4.5" x 3"H	Nylon
□910145	22" x 18" x 4.5" x 3"H	□910155	26" x 18" x 4.5" x 3"H	Nylon

Purpose

Skil-Care's Bariatric Foam Cushion is designed to reduce sitting pressure and control forward sliding.

Cover Straps

Covers have hook and loop straps that secure the cushion to the wheelchair.

WARNINGS



- Do not expose wedge to open flames, burning cigarettes, and other sources of ignition. If ignited, wedge will burn rapidly and give off toxic fumes.
- Remove foam pad from cover before laundering cover. Do not wash foam. Laundering will cause the foam to lose its fire retardant characteristics.

Cover Care

Nylon cover may be laundered and dried at temperatures that do not exceed 180°F (82°C). Do not use bleach. Remove foam from cover before laundering. Laundering foam will cause the foam to lose its fire retardant characteristics.

Skil-Care's Bariatric Foam Cushion meets California 117B flammability standards.