
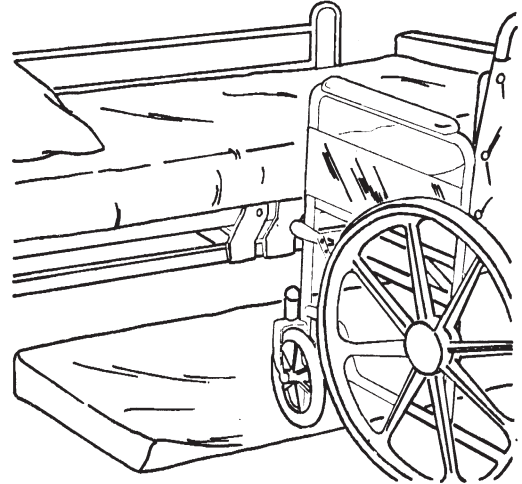


Roll-On Bedside Mat

Reorder #	Description	Size	Unit
911548	Roll-On Bedside Mat	68" x 26" x 1.5"	Each
911549	Roll-On Bedside Mat	68" x 36" x 1.5"	Each

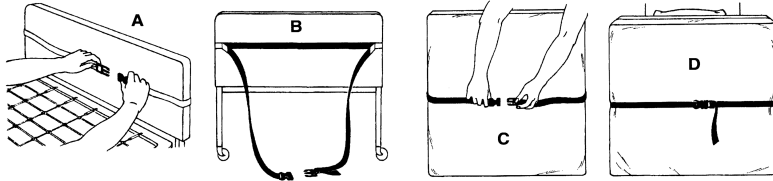

PURPOSE

Placed alongside a bed, the Roll-On Bedside Mat reduces the impact of a fall from bed. The tapered edges are designed for easy wheelchair roll-on and are safer for residents who might stumble on standard floor mats.

INSTRUCTIONS

Place Roll-On Bedside Mat on the floor next to the bed. Two mats will be required if side rails on both sides of the bed are lowered. Important: Place the Roll-On Bedside Mat on the floor with the label reading THIS SIDE UP facing up.

Important Notices

1. It is the responsibility of each facility's appropriate medical professionals to determine if the Roll-On Bedside Mat is suitable as an injury-prevention device for a specific patient.
2. To prevent staff members from stumbling, make certain that all staff members, especially those working at night, are aware that the Roll-On Bedside Mat has been placed alongside a bed.

Cleaning Instructions

Comply with OSHA regulations and CDC recommendations relating to blood borne pathogens. Follow accepted precautions. Use an EPA-registered disinfectant according to the manufacturer's instructions when performing disinfecting procedures.

Disinfection

1. Before disinfection, clean the cover surface using a damp cloth or sponge and mild cleanser.
2. Disinfect the cover surface using a appropriate EPA-registered disinfectant and follow manufacturer's instructions. Do not saturate the cover or allow cleaning solutions to pool on the cover surface.

Infection

Inspect the cover before each use. Do not use if the cover has cuts, rips, cracks, or tears.