





Re-Order #	Description	Size	U/M
911570	Folding Mat	.75" x 24" x 70"	Each
911575	Non-Folding Mat	.75" x 24" x 70"	Each





Purpose

When bed rails cannot be used, CMS suggests, "Placing the bed lower to the floor and surrounding the bed with a soft mat." Skil-Care's Safe & Sound Fall Mat is designed to reduce the impact of a fall from bed. It features a low-profile/tapered-edge design that permits easy wheelchair roll on and reduces the likelihood of a resident or caregiver from stumbling as he/she approaches the mat.

Easy to Use

Place the fall mat on the floor adjacent to the bed, aligning it alongside the bed. Assure that there aren't any gaps between the bed and mat. The blue side of the mat with the logo faces up. In the event that the folding mat is used, undo the strap and unfold the mat prior to placing on the floor.

After usage, when stored, the folding mat can be folded and stored at the foot of the bed. Note, that the tie-strap is provided that allows attachment to the bed.

Features and Benefits

- The beveled edge permits carts, chairs and IV stands to roll over easily and will not impede walking activities or patient stability.
- The mat is covered with an easy to clean, water resistant PVC material with anti-bacterial protection.
- The inside contains a unique polyurethane foam material that provides comfort and energy absorption to reduce impacts from accidental falls.
- The anti-slide bottom keeps the mat stable on the floor and the non-skid surface prevents the patient or caregiver from sliding.
- The mat meets stringent FRCFR1630 fire code rating.
- This easy-to-clean mat can be mopped with typical hospital floor cleaning products.
- The mat is latex free.
- Folding model comes with straps that allows it to be attached to the foot board above the floor.
- Available in two styles; folding and non-folding.

Important Notice

- These fall mats significantly reduce the impact when a resident falls on them but there is no guarantee that no injury will result in a fall. Injury is dependent on height, body shape, direction of fall, angle point of body contact and patients health and strength
- It is suggested that all staff members be-informed that a fall mat is in place to prevent tripping
- Since the floor generally contains significant exposure to bacteria, it is strongly recommended that the folding mat be stored above the floor, on the head board and the non-folding mat be stored in an area not exposed to housekeeping when they wash the floors.

Easy Care

The Safe & Sound Fall Mat could be wiped clean with a spray and wipe cleanser. If disinfected, avoid using chlorine (bleach) chemicals.

Extended Guarantee

Since this mat is extremely durable and resistant to tears, this product will have an extended guarantee period of 18 months.

