



ReOrder #	Description	Size	Unit
912420	Sensory Stimulation Gel Pad	7" x 16"	Each


### Purpose

Skil-Care's Sensory Stimulator Gel Pad contains water-based gel and colorful stars that move through the gel as the pad is handled or touched. It is intended to calm agitated, diminished-capacity individuals by providing tactile and visual stimulation.

### Easy to Use

The gel pad may be placed on a table top or given to an individual seated in a chair. Its soothing effect may decrease restlessness for some diminished-capacity individuals, thereby reducing wandering or chair exits.

The Sensory Stimulator Gel Pad may be heated in warm water to sooth arthritic fingers. Do not heat the gel pad beyond 110°F (43°C). Do not place the gel pad in a microwave.

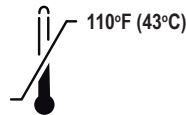

### Easy Care

The gel pad may be cleaned with a mild, non-chlorinated detergent and can be disinfected with a mild, non-chloride-containing disinfectant. Although the gel is not toxic, medical assistance must be invoked if the gel is taken through the mouth. If the product is cracked or damaged, it must be discarded.