
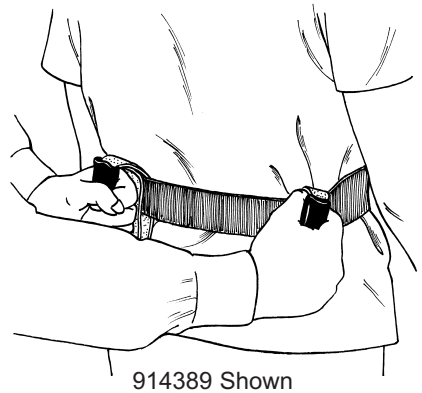


Re-order #	Description	Size	Wt.	Unit
☐ 913145	Nylon Webbing, Blue	5"L	.3 lbs	6/pc
☐ 914389	Delux rubber grip/1" Plastic Webbing, White	5"L	.5 lbs	6/pc

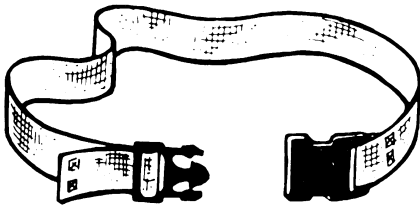

PURPOSE

Skil-Care's gait belt handles are designed to provide caregivers with a more secure and comfortable way to grip gait belts when a patient is being ambulated. For general application the model #913145 could be used but for a more pathogen free environment, model #914389 should be considered, especially when used with the Skil-Care Pathoshield® Wipe-clean Plastic gait belts.

USE INSTRUCTIONS

It is recommend that two to three handles be placed on the gait belt by placing the gait belt through the outer (smaller loop). If more than one care giver is ambulating the resident it is recommended that a trial walk be made with an ambulatory person so that ambulation is coordinated. The orientation of the handles should be made to maximize stability of the patient and care givers. See illustration for a possible location of the handles.

CARE INSTRUCTIONS

Both models can be wiped clean with a mild disinfectant.