

Purpose

Skil-Care's Gait Belt Loop Handles are designed to provide caregivers with a more secure and comfortable way to grip the gait belt when a patient is being ambulated.

Re-Order #	Description	Size	Unit
913145	Gait Belt Loop Handles	5"L	6/pk
914389	Delux rubber grip - 1" Plastic Webbing	5"L	6/pk

Instructions

1. Place two loops handles on each gait belt. The belt should be inserted through the smaller, inside loop.
The larger, outside loop is the grip handle.
2. Secure gait belt around patient's waist and position one loop at the patient's back and the other at his/her side.
3. Right-handed caregivers should position the side loop at the patient's left side, and left-handed caregivers position the side loop at the patient's right side. The caregiver's strong hand should be used to grip the loop at the patient's back.

