



teres





Fig. 1

Re-Order #	Description	Size	U/M
914335	Face Mask with Double Strap	Universal	12/PK
914336	Face Mask with Ear Loop	Universal	12/PK

Purpose

These face mask covers the nose and mouth. The intended use is to provide a barrier to help shield the nose and mouth.

Easy to Use

Clean your hands with soap and water or hand sanitizer before touching the mask. Pick up the mask by the straps and make sure there are no obvious tears or holes in either side of the mask. Place the mask on your face with the fleece side facing the mouth. Secure the mask using the straps; For the ear loop style place the straps around the ear. For the double straps style place the straps around the back of the head. Ensure the mask covers your nose and mouth and is secure under the chin. Avoid touching your mask while wearing it.

Please note on the ear loop style in the event that the mask is too loose, the loops can be twisted in a figure eight as seen in Fig. 1.

Material

The face masks are manufactured from an absorbent fleece polyester that is launderable.

Caution

This product is <u>not</u> NIOSH or FDA approved for the purpose of protecting the user against entry of infectious aerosolized particles. This product is not sterilized.

Cleaning Instructions

The nose and mouth mask can be laundered at temperatures below 180°F (82°C).

