


Wheelchair Chart/ Universal Holder

Reorder #	Description	Size	Wt.	Unit
□914341	Vinyl Bag w/ 5 x 6½" See thru window	15.5"W x 14"H	.04 lbs	Ea

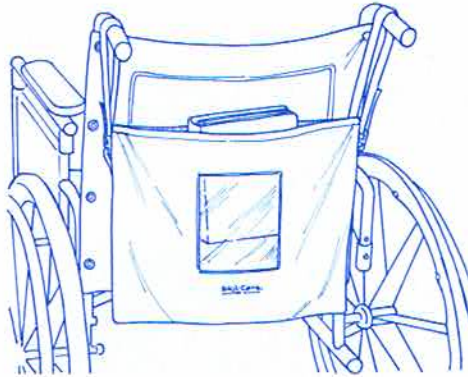

PURPOSE

Provides a safe and clean holder to house patients medical charts. The convenient 5"x6.5" window provides the opportunity to view the chart without the need to remove the chart. This convenient bag can also be used for the patients personal items when not used as a chart holder.

USE INSTRUCTIONS

Place the chart holder with the window toward the outside of the wheelchair, by placing the respective straps around either wheelchair hand grip. If necessary, the straps can be adjusted with the Velcro straps.

CARE INSTRUCTIONS

The Universal holder is made from reinforced vinyl and can be cleaned with a mild detergent. Do not launder this item.