



Re-Order #	Description	Size	Unit
914351	Seat Buddy - Polyester Cover	16" x 16" x 1.5	Each
914350	Seat Buddy - Blue Cozy Cloth Cover	16" x 16" x 1.5	Each
914352	Seat Buddy - Grey Cozy Cloth Cover	16" x 16" x 1.5	Each
914353	Seat Buddy - Pink Cozy Cloth Cover	16" x 16" x 1.5	Each

Thank you for choosing the Skil-Care™ Seat Buddy Cushion. This portable, versatile cushion is designed to provide added comfort and stability for everyday seating from wheelchairs and office chairs to school desks, casinos, restaurants, stadium seats, travel and beyond. Its non-skid bottom ensures safety, while the snap-close design makes it easy to carry anywhere.

Easy to Use

1. Place the Seat Buddy Cushion on the chair, wheelchair, or seating surface with the non-skid side facing down.
2. Sit centered on the cushion for best comfort and support.
3. After use, fold the cushion in half and snap it closed for carrying or storage.
4. Carry the cushion using the snap-closed handle or place it in a bag.

Cleaning

Wipe the cushion cover with a damp cloth and mild soap as needed. You may also remove the outer zipper cover and launder according to the care label. Ensure cushion is fully dry before reuse. Do not bleach or use harsh chemicals.

Caution

- Always place the cushion on a stable, flat seating surface.
- Ensure the non-skid bottom is facing down before use.
- Do not stand on or use the cushion as a step stool.
- Inspect regularly for wear and replace if damaged.

Warranty

The Skil-Care™ Seat Buddy Cushion is backed by a one-year limited warranty against defects in materials and workmanship. For warranty claims, please contact Skil-Care™ customer service with proof of purchase.