

Re-Order #	Description	Size	Wt.	U/M
914376	Yellow	2" x 60"	350 lbs.	Each
914377	Yellow	2" x 72"		Each
914378	Green	2" x 60"		Each
914379	Green	2" x 72"		Each
914380	Blue	2" x 60"		Each
914382	Blue	2" x 72"		Each
914384	Stars/Stripes	2" x 60"		Each
914386	Stars/Stripes	2" x 72"		Each
914387	Orange	2" x 60"		Each
914388	Orange	2" x 72"		Each



**Purpose**

The Gait Belt is intended for use when lifting, ambulating or transferring patients. It is not to be used as a safety restraint in a chair or bed. Since the Bio-Shield Gait Belts minimize the risk of harboring pathogens, they are to be used in lieu of fabric gait belts when a reduced pathogen environment is desired.

**Easy to Use**

When an individual is at risk in ambulating without assistance gait belts should be used. The gait belt should be placed around the waist and when the buckle is closed the gait belt should fit snugly but not too tight to introduce patient discomfort.

The Delrin side release buckles are secured by inserting the male end of the buckle into the female side. To release the buckle, simultaneously press the tabs at the top and bottom of the buckle. Before ambulating the patient make certain that the buckle is positioned out of reach of the patient.

The patient should be ambulated by having the caregiver placing their hands through the belt palms up and grasping the belt. Before ambulating the resident, it should be confirmed that the number of caregiver's and their ability is sufficient for ambulation without introducing risk to either the caregiver or patient. The belt should be inspected before each use to assure that it isn't defective.

**Easy Care**

These devices may be wiped clean with a mild disinfectant.

**Bio-Shield Gait Belt Handles**

**Purpose**


The handles are designed to aid caregivers by providing an ergonomically secure and comfortable way to grip the gait belt when assisting with standing and walking. The handle fits around the female portion of the gait belt buckle.

**Easy to Use**

1. Place the female portion of the gait belt buckle through the smaller, inside loop portion of the handle. The larger, outside loop is the grip handle.

**Easy Care**

These devices may be wiped clean with a mild disinfectant.

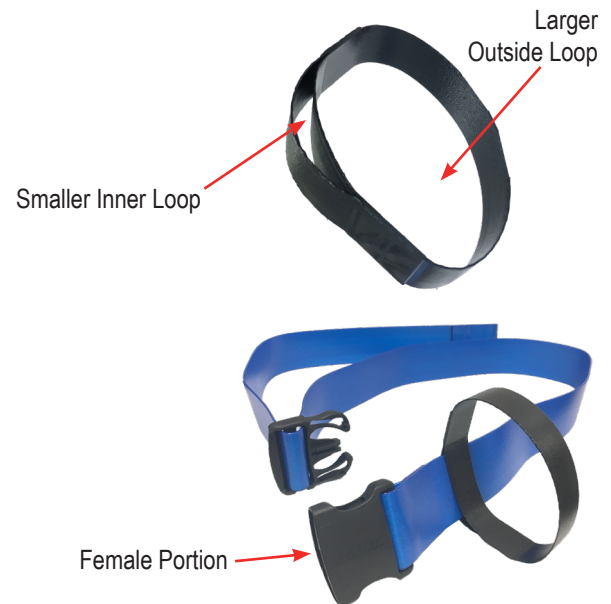

Re-Order #	Description	Size	Unit
913150	Bio-Shield Gait Belt Handle	5"L	Pair