



- Box Dimensions: 17" x 17" x 4"
- Illuminated Area: 15" x 15"
- 6 foot, 115VAC electric cord

Reorder#	Description	Unit
914514	Light Box	Each

The light box features a 15" x 15" light screen with a raised edge to prevent lighted objects from falling off. The illuminated surface can also be used to backlight translucent gel shapes that provide visual and tactile stimulation for children and adults. It also provides a surface for artists, photographers and children to illuminate drawings, transparent objects for the purpose of viewing or tracing. Numerous optional gel pads are available from Skil-Care Corporation that are designed to be used with the light box.


Cautions:

- The unit may be cleaned, with a damp cloth that contains soap or any cleaner. Do not spray any liquid directly on the product.
- Do not attempt to dismantle product, call customer care at 800-431-2972 if there are any issues with product.
- This product can be used by any child. However it should not be used without proper supervision of a parent or professional who can assess the abilities and limitations of the user.

Operation:

- Make sure on-off switch is in OFF position.
- Plug unit into the wall outlet.
- Turn unit to "Steady On", unit will light up continuously.
- Turn unit off.
- Turn unit on to "Press On", then press top of unit for it to light. In this mode the unit will light up only when pressed.
- Turn unit off
- Turn to off position when unit is not in use.
- When ready to use, place gel pads or item to be traced on the unit and set the switch to the desired position.

See other side for additional Light Box Products

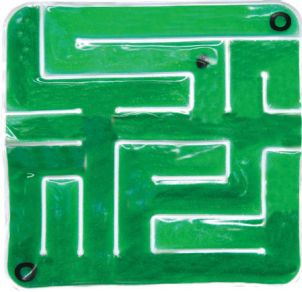


Clear Gel Maze

ReOrder #	Description	Size	U/M
914517	Gel-Maze w/marbles Clear Gel	14" x 14" x 1"	Ea.



Spiral Gel Pad

ReOrder #	Description	Size	U/M
912441	Spiral Gel Pad	15" x 15"	Ea.

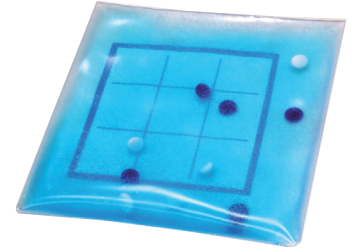


Tic-Tac-Toe Gel Pad

ReOrder #	Description	Size	U/M
914516	Tic-Tac-Toe Gel Pad (Clear Gel)	10" x 10"	Ea.


Gel Maze Series



ReOrder #	Description	Colors	Size	U/M
912442B	Quad Tree Gel Pad	Blue	15" x 15"	Ea.
912442G	Quad Tree Gel Pad	Green	15" x 15"	Ea.
912442C	Quad Tree Gel Pad	Clear	15" x 15"	Ea.
912442R	Quad Tree Gel Pad	Red	15" x 15"	Ea.
912442Y	Quad Tree Gel Pad	Yellow	15" x 15"	Ea.

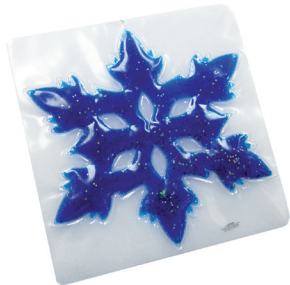

ReOrder #	Description	Colors	Size	U/M
912447B	Snow Flake Gel Pad	Blue	15" x 15"	Ea.
912447G	Snow Flake Gel Pad	Green	15" x 15"	Ea.
912447C	Snow Flake Gel Pad	Clear	15" x 15"	Ea.
912447R	Snow Flake Gel Pad	Red	15" x 15"	Ea.
912447Y	Snow Flake Gel Pad	Yellow	15" x 15"	Ea.

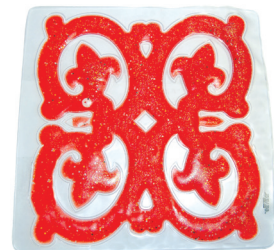

ReOrder #	Description	Colors	Size	U/M
912445B	Quad Spade Gel Pad	Blue	15" x 15"	Ea.
912445G	Quad Spade Gel Pad	Green	15" x 15"	Ea.
912445C	Quad Spade Gel Pad	Clear	15" x 15"	Ea.
912445R	Quad Spade Gel Pad	Red	15" x 15"	Ea.
912445Y	Quad Spade Gel Pad	Yellow	15" x 15"	Ea.



ReOrder #	Description	Colors	Size	U/M
912449B	8 Spoke Snow Flake Gel Pad	Blue	15" x 15"	Ea.
912449G	8 Spoke Snow Flake Gel Pad	Green	15" x 15"	Ea.
912449C	8 Spoke Snow Flake Gel Pad	Clear	15" x 15"	Ea.
912449R	8 Spoke Snow Flake Gel Pad	Red	15" x 15"	Ea.
912449Y	8 Spoke Snow Flake Gel Pad	Yellow	15" x 15"	Ea.

Clear Gel Shapes



914518



914513

ReOrder #	Description	Size	U/M
914518	Clear Geometric Gel Shapes (Circle, Triangle, Square, Rectangle and Hexagon)	5" max	Set
914513	Clear Gel Shapes (Pine Tree, Heart, Star, Flower and Moon)	5" max	Set