



914513 - Clear Gel Shapes
(LightBox not included)

Re-Order #	Description	Size	Units
914513	Clear Gel Shapes (Pine Tree, Heart, Flower, Moon, Star)	5" max.	Set
914519	Gel Shapes (Pine Tree, Heart, Flower, Moon, Star)	5" max.	Set

Purpose

It has been found that therapeutic touch (Krieger, 1997) is effective in decreasing anxiety. These multi-model products address sight and touch and are effective in reducing boredom and also serve as a tool in training for eye-hand coordination and also to improve finger strength and dexterity. The products can be used for color recognition, finger activities and shape recognition.

Easy to Use

The shapes should be placed on a surface convenient to the child. They should be encouraged to do finger exercises with the gel, place the shapes in various arrangements and also can be placed on the floor to create an obstacle course. The gel pads can be placed in warm water or in the refrigerator to enhance recognition of hot or cold.

Easy Care

The vinyl pads are heat sealed and can be cleaned with a mild detergent. Although the gel is nontoxic, medical care should be consulted if the gel is ingested. If a tear or rip is observed, the item should be discarded.

Caution

The product must always be used under the supervision of a care-giver. We strongly recommend that this product not be used with individuals who are under 5 years cognitively. If cooling the product, place it in a refrigerator and not the freezer and conversely if heating the product, place it in the water at a temperature that will not cause skin damage.

Manufactured by: Skil-Care Corporation • Made in the U.S.A