



914752



914750

Re-Order #	Description	Size	Unit
914750	Crash Pad TM	3' x 4'	Each
914752	Crash Pad TM	5' x 5'	Each
914756	Optional Nylon Outer Cover	3' x 4'	Each
914758	Optional Nylon Outer Cover	5' x 5'	Each
914757	Optional Cozy Cloth Outer Cover	3' x 4'	Each
914759	Optional Cozy Cloth Outer Cover	5' x 5'	Each

Purpose

This unique "Crash PadTM" was developed primarily for young adults to serve as a fall pad whether for exercises or learning appropriate responses to falling. In many situations, children with developmental delay or sensory integration disorder need to learn the appropriate falling procedures. Associated with a primitive reflex called the moro reflex.

Easy to Use

To economize in shipping cost the Crash Pads are compressed prior to shipping. When they are opened they generally will return to about 80% of their full volume in about 8 hours and will be fully expanded after three days. Although the nylon outer covering can be easily cleaned, it is impractical to remove the foam if it is desired to launder the cover. It is strongly suggested that an optional cover that fits over the inner cover be used. These optional covers come in two fabrics, Cozy Cloth and Nylon and can be easily removed for laundering.

Caution

When the pad is resting there is a quantity of entrapped air and when you jump on the pad there is a slight bounce effect. If an additional outer cover is used, it is critical to align the vent holes by lining up the red webbing to allow the air to escape more easily to minimize this "bounce effect". Do not use this crash pad within 5 feet from a wall. Since jumping, flipping or bouncing involves some risk, it is mandatory that these Crash Pads be used with a trained professional in attendance.

Easy Care

The inner cover can be wiped clean with a standard cleaning solution. The optional outer cover can be laundered at temperatures below 180°F (82°C)