


## Hidden Letter Glitter Shapes

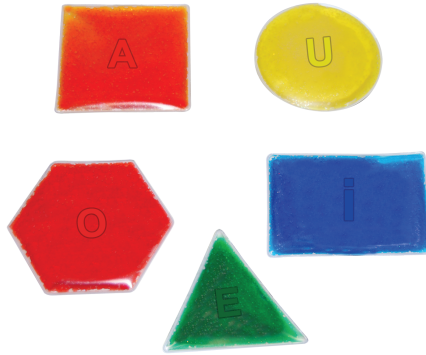

Reorder #	Description	Size	Wt. (lbs)	Unit
□914615	Color Glitter Shapes with A.E.I.O.U	5" max	1.2 lbs	5/pk
□914616	Color Glitter Shapes with R.S.T.L.N	5" max	1.2 lbs	5/pk

### PURPOSE

Skil-Care Hidden Glitter Gel Shapes are filled with non-toxic gel that stimulates the visual and tactile systems. These pads can be used to assist with visual perception, pre-writing skills, letter and number recognition and tactile awareness.

### USE INSTRUCTIONS

These pads are to be placed on a table or other convenient height. The client should be instructed to identify or trace numbers or letters with index finger. The pads could also be used for color, shape and vowel recognition games. All of these products should be used under direction of a care giver.

### CARE INSTRUCTIONS

These vinyl products could be cleaned with a mild detergent. Although the gel is non-toxic, medical care should be consulted if the gel is ingested. If a tear or rip is observed in the gel the unit should be discarded. These products must always be used under the direct supervision of a care giver.

Manufactured by: Skil-Care Corporation • Made in the U.S.A