Gel-Foam Picture Frame



Reorder #	Description	Size	Wt. (lbs)	Unit
914722	Gel Foam Picture Frame	10"x 17"	2.1 lbs	Fa

PURPOSE

These multi-modal gel/foam picture frames address sight and touch and are effective in reducing anxiety, boredom and also serve as a tool in training for eyehand coordination and also improve finger strength and dexterity.

USE INSTRUCTIONS

Have the person or instructor prepare a visual array of figures, numbers or letters on a stiff piece of paper or cardboard that fits into the pocket. To enhance the visual and touch experience, insert the prepared art into the open pocket with the artwork facing the gel. Have the person identify objects or trace the object/alpha numeric through the gel. This is a very useful exercise especially for pre-handwriting activities and for cognitive association activities.

CARE INSTRUCTIONS

The vinyl Gel Foam Picture Frames are heat sealed and can be cleaned with a mild, non-chlorinated detergent and must be disinfected with a mild, non-chloride-containing disinfectant. Although the gel is non-toxic, medical care should be consulted if the gel is ingested. If a tear or rip is observed in the gel the unit should be discarded. These products must always be used under the direct supervision of a care giver.

Manufactured by: Skil-Care Corporation • Made in the U.S.A

