



Re-Order #	Description	Size	Unit
914750	Crash Pad™	3' x 4'	Each
914752	Crash Pad™	5' x 5'	Each
914751	Crash Pad™ with extra nylon cover	3' x 4'	Each
914753	Crash Pad™ with extra nylon cover	5' x 5'	Each
914754	Crash Pad™ with extra cozy cloth cover	3' x 4'	Each
914755	Crash Pad™ with extra cozy cloth cover	5' x 5'	Each



Purpose

The Crash Pad is a soft-landing zone for kids requiring sensory input or can be used as a giant bean bag for reading & relaxing. Ideal for sensory rooms in schools, clinics, or at home. This pad is filled with soft foam pieces and comes with a durable water resistant anti-bacterial nylon cover.

Easy to Use

This item ships compressed and will expand to complete fullness within 48 hours. The mesh vents gently release air during use and allow the pad to return to its natural ideal state. To ensure safe crashing, place the pad away from walls and furniture. Do not block mesh vents. The nylon cover easily wipes clean and can be laundered separately. Do not launder the foam pieces.

Caution

The pad is free of flame retardant properties. This product must always be used under the direct supervision of an adult or care giver. The zipper can be opened and the foam pieces should not be injected.

Easy Care

The nylon cover (without the foam pieces) can be laundered at temperatures below 180°F (82°C). However we do not recommend laundering the inner cover, it should only be wipe cleaned. An optional replacement cover is available and can be easily slipped over the inner cover.



Re-Order	Description	Size	Units
914780	Outer Cover for Crash Pad™ - Green Nylon	3' x 4'	Each
914782	Outer Cover for Crash Pad™ - Blue Nylon	5' x 5'	Each
914781	Cozy Cloth Outer Cover for Crash Pad™	3' x 4'	Each
914783	Cozy Cloth Outer Cover for Crash Pad™	5' x 5'	Each

Purpose

The Crash Pad™ is filled with foam and contained by an outer nylon cover. Although this cover can be wiped clean if one wants to launder the cover, it is to time consuming to remove and replace the foam. For this reason, an optional outer cover is available that fits over the inner cover which is now available in two fabrics. This cover can be easily removed and laundered.

In order to minimize the “bounce effect” due to trapped air, vent holes are placed on the inner and outer cover.

Easy to Use

When the outer cover is placed over the inner cover it is critical that the vent holes are aligned to allow for the air to escape quickly. For this reason, a red strip of webbing is placed on both covers and when placing the 2nd cover on, assure that the red webbing is aligned in the same direction as the inner cover (Nylon Outer Cover Only).

Easy Care

Both covers can be cleaned with a germicidal cleaning solution or soap and water. The outer cover can be laundered at temperatures below 180° F (82° C). Dry at the low setting. Do not use bleach directly on the fabric.